- Ⓛ Gebrauchsanweisung
- Ⓖ Instructions for use
- Ⓕ Mode d'emploi
- Ⓔ Instrucciones para el uso
- Ⓘ Istruzioni per l'uso
- Ⓜ Gebruiksaanwijzing

- Ⓟ Instruções de utilização
- Ⓛ Brugsanvisning
- Ⓢ Bruksanvisning
- Ⓟ Instrukcja obsługi
- Ⓖ Οδηγίες χρήσης
- Ⓒ Návod k použití

- Ⓛ Navodilo za uporabo
- Ⓢ Návod na použitie
- Ⓜ Способ применения

Maximex®

Ⓜ MOORWÄRMER

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Anwendung:

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For tense muscles or as a bed warmer

Application:

Since the actual performance of various microwave oven manufacturers with the same specification is different, we recommend testing the optimal warm-up time. Place the moor warmer flat into the microwave oven and distribute the gel evenly. Now turn on the microwave oven for 2 minutes in the thawing function. Afterwards, remove the warmer and knead it well so that the warmth is evenly distributed. Repeat this procedure until the desired temperature is reached. Caution! Overheating can destroy the moor warmer.

Attention:

Do not heat in an ordinary oven or grill.

Caution:

Overheating can destroy the moor warmer. Leaky or empty moor warmers must not be heated.

Care instructions:

In the event of soiling, the moor warmer can be cleaned under running water at max. 30 °C. The use of scouring agents will damage the surface. Initially, the velour material will shed some fluff. Therefore brush the velour with a clothes brush prior to use. Any fluff sticking to your skin and clothing may easily be removed with a towel or clothes brush. The shedding of fluff will stop after a few applications.

Gel information:

The contents of the moor warmer are non-toxic. Should gel come into contact with the skin or eyes, simply wash off with plenty of water. There is no record of skin or eye irritations.

In order to ensure enjoyment of your new moor warmer for the longest possible amount of time, you should follow these instructions exactly!!!

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