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MedoNail

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MedoNail

Fungal Nail Treatment liquid

Read the instructions for use before using this medical device.

- To treat fungal nail infections on the hands and feet of adults.
- Proven to effectively treat fungi that cause fungal nails.*
- A unique mix containing ISK1320 and essential oils.
- Creates an unfavourable environment for fungi to grow and spread on the nail surface.

*Based on in vitro antimycotic effectiveness tests conducted on *Trichophyton rubrum*, *Trichophyton mentagrophytes*, *Epidermophyton floccosum* and *Candida albicans*.

General advice: Before using MedoNail, read the instructions carefully, even if you have used this medical device before. The information contained in a new pack may have been revised in order to reflect the latest scientific insights and data. Nevertheless, you should use MedoNail with care in order to obtain good results. Keep these instructions – you may need to refer to them during treatment.

Application and dosage (for adults only): MedoNail is a medical device that helps in the treatment of fungal nails (onychomycosis). The liquid treatment is for external use only.

Apply the product to the affected area twice or three times a day (mornings and evenings, for instance) until the fungal nail has fully grown out. MedoNail Fungal Nail Treatment liquid can be combined with the use of oral antimycotics.

To achieve faster, better results, carefully filing the fungal nail once a week is recommended. If treatment is stopped, the fungal nail will return more quickly. It is therefore important to continue using MedoNail until the fungal nail has gone.

As damp environments are one of the causes of fungal nail infections, it is strongly recommended that you wear socks made only of absorbent materials, such as cotton or wool. This is important, as moulds are commonly found in moist environments, where they can multiply. If you have a fungal nail, it is important that you keep your feet dry.

Mode of action: Thanks to a unique combination of ingredients, MedoNail Fungal Nail forms a layer on the nail plate, which creates a mildly acid environment unfavourable for the growth and spread of fungi.

Use during pregnancy and/or breastfeeding: No information is available regarding the use of MedoNail during pregnancy or the breastfeeding phase. If you are pregnant or breastfeeding, consult your doctor or pharmacist before using.

Side effects: Possible side effects includes: erythema, irritation, edema, pain, nail discoloration and onycholysis. Allergic reactions.

- Warnings:**
- For external use only.
 - Check the list of ingredients to know if you are allergic to any of the ingredients contained in the gel.
 - Avoid contact with eyes or mucous membranes.
 - Do not apply to open wounds or damaged skin.
 - Do not bandage tightly.
 - Do not apply heat to the area of application.
 - Pregnant or breastfeeding patients should seek advice of health professional before using MedoNail.
 - If you experience allergic reactions, stop using the product.
 - Keep out of reach of young children.
 - In case of no effect, consult a physician.

Ingredients: Aqua, Ethoxydiglycol, ISK1320, Pentylene Glycol, Alcohol Denat., Tridiceth-9, PEG-40, Hydrogenated Castor Oil, Benzyl Alcohol, Limonene, Glycerin, Natiolact, Propylene Glycol, PVP, Xanthan Gum, Eucalyptus Globulus Leaf Oil, Lavandula Hybrida Oil, Linalool, Dimethyl Isosorbide, Citrus Aurantium Dulcis Oil, Pogostemon Cablin Oil, Disodium Phosphate, Geraniol, Citral, Biotin, Coumarin, Citric Acid, Farnesol.

Contents: 10 ml
Storage instructions: Store in a dry and dark place at room temperature (5-25°C).

See the bottom of the pack for the expiry date.

Medical device: MedoNail Fungal nail treatment is a class IIa medical device according to European medical device directive no. 93/42/EEC.

Date on which these instructions were last revised:
September 2022 • Art.-No. EU-1915/M - 073460V1.3

Swiss Beauty AG, Walke 43, CH-9100 Herisau

Distribution for EU:
MedoVital Handels GmbH
Bayreutherstrasse 13
DE-67059 Ludwigshafen

Distribution for UK:
WFBM Healthcare Ltd.
Genesis centre
GB-Stoke-on-Trent ST6 4BF

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Peters Krizman AG