
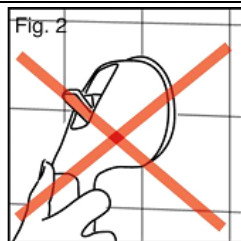
i Imported by - Importé par:
 Gd import (Genius Ideas & Welllys)
 190 chemin de la Frayère - 06530 Peymeinade - France
 www.gdimport.com
 www.genius-ideas.com
 www.wellys-care.com gdimport@gimport.com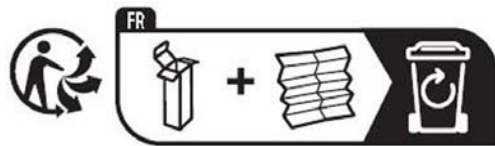

FR - POIGNEE DE BAIN VENTOUSES "GLOW" – 134562 – VERSION 2026-13-02

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GB - SUCTION CUP HANDLE "GLOW IN THE DARK" – 134562

Caution: This handle must be used as balancing assist only. It cannot support your full body weight. Before each use check that the grip is safely fixed and check the tightness periodically. As this product is a balancing assist and support item, we recommend to anyone having any difficulties to follow these instructions to ask help from a qualified person.

Note: For maximum support, this item must be used correctly. An incorrect use may cause instability of the handle or it might get detached. The suction cups must be placed on a completely smooth non-porous surface. Placing suction cups over cracks or grout lines between tiles will prevent them from holding properly and the item will be unstable (Fig. 2). Check that the support can stand this item (do not place on badly fixed or mobile tiles).

Use:

Clean and wet the surface before installing. The tilting levers must be up and loose. Press the suction cups onto the surface, all of them must be completely on the surface. Press each tilting lever down to fix the handle. Test the adhesion before each use. If needed clean the suction cups and install the handle again following the previous steps. Uninstall and reinstall every week.

The two glow-in-the-dark rings are charged by light and provide orientation in the dark.

Attention! When attaching the handle to the edge of the bathtub, make sure that the edge is wide enough so that both suction cups are level and completely, evenly seated.

Max weight (when pulling out): **100kg.**

THE PULLING MUST BE PERPENDICULAR TO THE GRIP AS SHOWN ON FIGURE 1.

DO NOT PULL STRAIGHT UP OR DOWN AS THIS MAY RESULT IN A FALL.

PUTTING THE GRIP ON A WET SURFACE MAY CAUSE SLIDING AND RELEASING THE GRIP.

CAUTION:

- Never use the suction grip on rough and/or porous surfaces, wood, painted walls or other finished surfaces. Never use the suction grip on moving surfaces. When placing the suction grip on not cleaned and not suitable surfaces, there is risk of injury!
- Do not use the suction grip when there are signs of damage.
- Never use sharp objects to release the suction cup from the surface.
- Only use the suction grip on smooth, level and clean surfaces such as tiles, glass, acrylic glass or fiberglass.
- Always check for the suction grip to be properly placed on the surface, and before usage check for the locking lever to be safely in the "locked position".
- Do not leave the suction grip fixed on a surface for an extended period of time. If left fixed, the suction strength will deteriorate.

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