Imported by - Importé par:

GD import (Genius Ideas & Welllys)

190 chemin de la Frayère - 06530 Peymeinade - France

[www.gdimport.com](http://www.gdimport.com)

[www.genius-ideas.com](http://www.genius-ideas.com)

[www.wellys-care.com](http://www.wellys-care.com)

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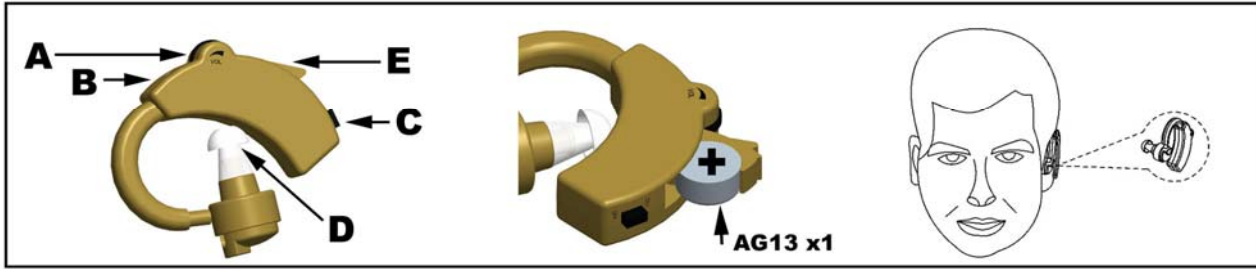
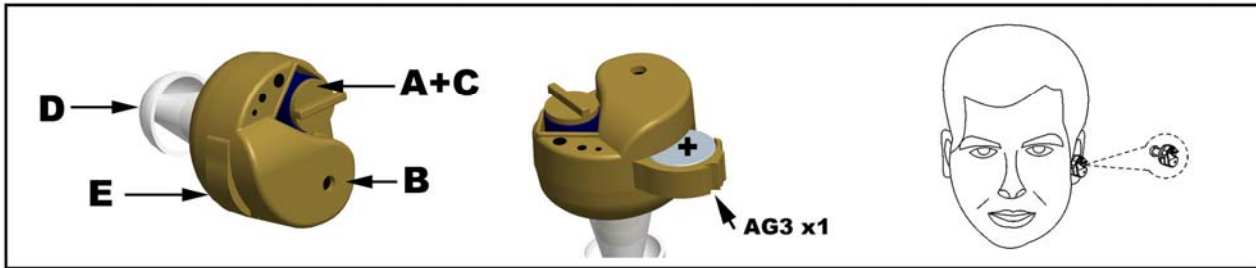
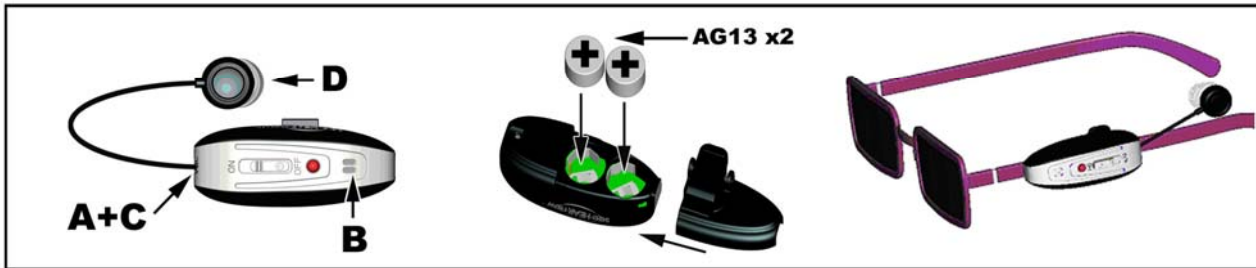
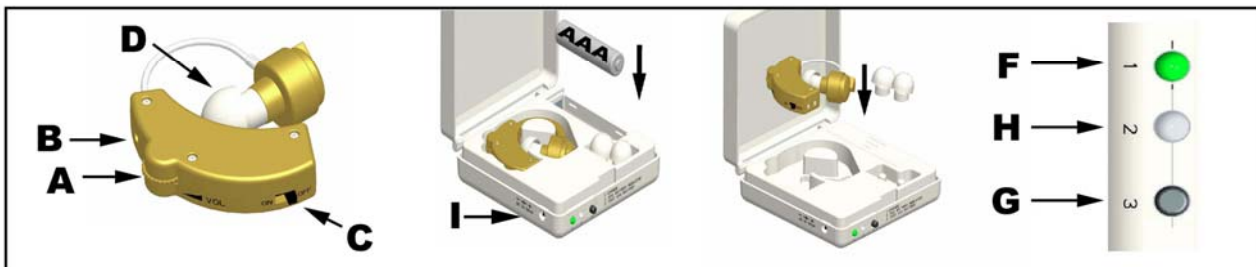
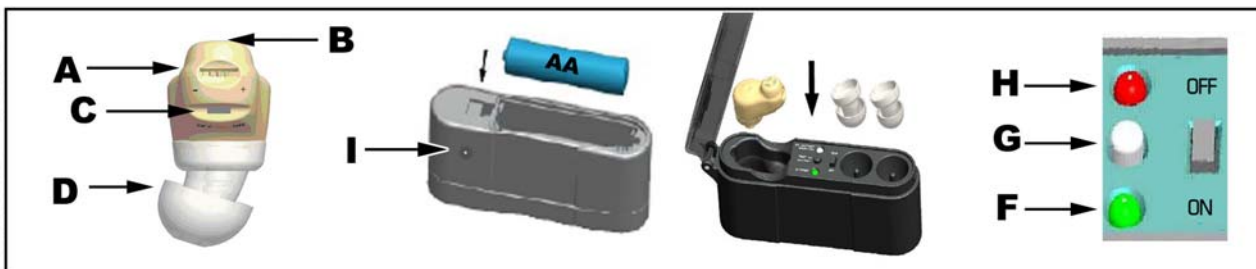
**Please carefully read through the operating instructions prior to using the device for the first time and store these instructions in a safe place. The instructions are to accompany the device if it is passed on to others. This device is not designed to be used by persons (including children) with impaired physical, sensory or mental abilities or those lacking experience and/or knowledge unless they are supervised by a person responsible for their safety or received instructions on how to use the device from them.**

Sound Zoomers allow to optimize the listening. More or less miniaturized, they are used to filter the sound to make it clearer with a deep effect. With their adjustable volume, they offer an upper comfort to follow a conversation, listen to a program broadcast with a minimized volume and to hear the sounds of nature.

### **User Instructions:**

1. Put the silicone tip wished on the sound zoomer.
  2. Turn the volume button in its minimum or switch off.
  3. Verify that the button battery (batteries) is (are) well positioned.
  4. Insert the silicone tip into the ear by making sure that the sound zoomer is well placed on the ear (see images).
  5. Switch-on the button of the volume.
  6. Increase gradually the volume button until you obtain a satisfactory result.
- Remove the sound zoomer in case of larsen (treble sound) and start again at step 2.

**Pictures details: A: Volume B: Microphone C: Switch D: Silicone tips E: Battery slot F Charging light G: Battery test button H: Test led I: Charger plug**

### **Sound zoomers working with button batteries (A, B and C Models):**

A, B and C Sound zoomers work with button batteries. Insert the batteries by respecting the -/+ side (see images). If the quality declines or if there is no more sound, change the batteries with an identical and new model. Remove the batteries if the sound zoomer may not be used for 1 month.

### **“Rechargeable” sound zoomers with their battery charging base (D and F Models):**

D and F Sound zoomers can be recharged by being inserted into their cases. These cases work with sticks batteries “alkaline type” (see the plan). If it doesn't charge, make sure that the contacts on the deck and the sound zoomer are clean and not oxidized. If it doesn't charge, change the battery with an identical and new model. Remove the batteries if the sound zoomer may not be used for 1 month.

### **“Rechargeable” sound zoomers with AC adaptor (E Model):**

E Sound zoomer can be charged with its AC adaptor. The first charge takes about 5 hours. If the quality declines or if there is no more sound, charge the sound zoomer for 8 hours.

### **Optional AC adaptor (D and F Models):**

The charging cases can be fed by an optional AC adaptor. The complete charge takes about 3 hours. This charger is maybe already included in your pack. Ask for your nearest dealer at [info@wellys-care.com](mailto:info@wellys-care.com).

### **Maintenance:**

Charger and main unit: Clean with a soft and wet cloth. Make sure it is entirely dry before re-connecting the charger on sector (risk of electric shock). Silicone Tips: Clean with warm and soapy water, then rinse and dry carefully.

### **Caution:**

1. This product is not waterproof. Consequently, the user has to pay attention not to let any water penetrate in it (including the rain). If some water penetrates into the sound zoomer, remove the button battery and place the product in a warm and dry place. The consequences of a water exposure are not included in the guarantee.
2. It is important to increase the volume very gradually to avoid a rough increase of the flow.
3. This product must be used in a punctual way (maximum 2 hours) and with a reasonable volume.
4. This product is intended for an indoor or outdoor use with a quiet environment (40db to 60db). It is forbidden to use the device near a noisy source exceeding 70db (ex: Hoover, television with high volume, very crowded street).
5. The normal function of the product may be disturbed by Strong Electro Magnetic Interference if under EMC testing condition of: 80-2700MHz, with 3V/m at 1kHz modulation. If so, simply reset the product to resume normal operation by following the instruction manual. In case the function could not be resumed, please keep away the interference area.

### **Warranty:**

This product is guarantee against any manufacturing defect for a period of 1 year. In case of after sales problems, contact the distributor.

This product doesn't repair any ear problem and is not suited for a person suffering of a medium or high audition problem. If this is your case, you should return this article and consult a hearing specialist who will provide you an appropriate and custom-made solution. This product is only intended to improve the tone of a discussion or a TV program by bringing a different listening with an appreciably amplified volume (max. +20 / 25db).

**LIKE A MUSIC PLAYER:** A FULL POWER USE, A NOISY ENVIRONMENT (MORE THAN 70db), OR A TOO LONG LISTENING EXPOSURE CAN DAMAGE THE USER'S EAR. USE WITH A MODERATED VOLUME FOR MAXIMUM 2 HOURS.

## **DE - Super-Hörverstärker (MODELLE 008511 A, 007720 B, 008580 C, 008515 D, 008600 E 1 008531 F)**

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GB - This product is labelled in accordance with European Directive 2012/19/EU concerning used electrical and electronic appliances (WEEE). Do not dispose of electrical appliances as unsorted municipal waste, use separate collection facilities. Contact your local government for information regarding the collection systems available. If electrical appliances are disposed of in landfills or dumps, hazardous substances can leak into the groundwater and get into the food chain, damaging your health and well-being.

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GB - Batteries and storage batteries must not be disposed of in normal household waste. As the end user you are legally responsible for returning or properly disposing of the batteries. After use you can return the batteries to the place where you bought them free of charge (e. g. at the collection point in the shop or at local authority collection points).

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