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86kPa - 106kPa

EN Users Manual HAPI-90D

1. Selecting custom-fit Ear Bud. The Hearing aid comes with 2 different size of ear buds. Before use, choose from either L or M size to attach to the ear piece as shown in Pic#1. It is important to choose the appropriate ear bud size for maximum comfort and sound quality

• **Caution: Never insert the Ear Bud alone in your ear, it may be difficult to remove from the ear if it is not attached in the hearing aid.**

2. Inserting Battery

- Open the battery door in a direction as shown in Pic#2

- Insert battery, remove the tab before Zinc-Air battery Pic#3 * Close the battery door

3. The hearing aid is triggered "ON" as soon as you closed the battery door, and volume is present level 4 as shown in Pic#4. To attain desired listening volume, 7 - level selection from 1 to 7 could be adjusted. Pic#5

4. Wear the hearing aid in your ear by placing the Ear Bud in the ear. There is no need to push it strongly into your ear. The hearing aid is for LEFT and RIGHT ears by turning the ear piece as shown in Pic#6

5. Turn the unit "OFF" by pressing BOTH "Volume+" and "Volume-" buttons. The Hearing Aid will be "REACTIVE" back to last Volume Level by either pressing "Volume+" or "Volume-" as shown in Pic#7.

Cleaning your Hearing Aid. We recommend that you clean your hearing aid regularly (e.g. twice a week) to remove any earwax that may build up a result of regular use with a cotton tip or brush.

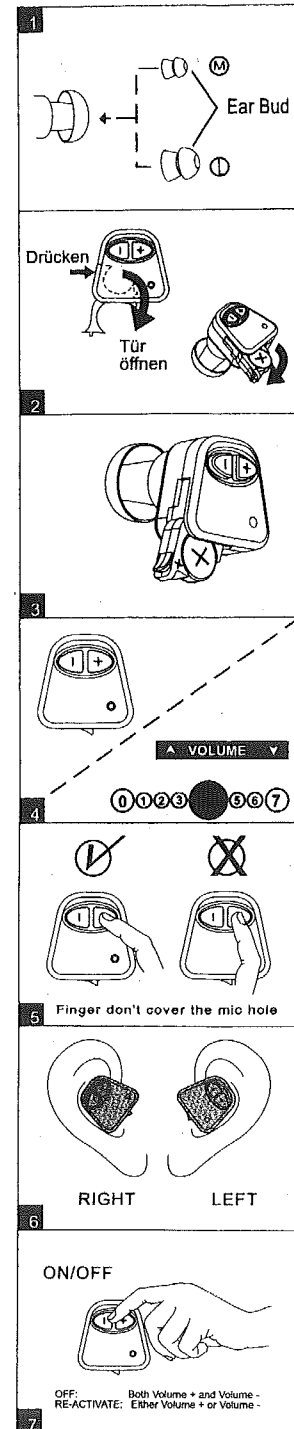
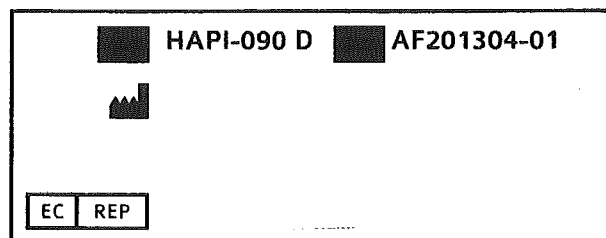
Caution: The hearing aid is not water proof product. Never immerse it completely in watery* IEC 529 (ECC) level 0>. It is suggested to place the hearing aid back to the carrying case after daily usage and well store it at room temperature with good ventilation and humidity<30%, and avoid contact with corrosive gases.

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