24779

Soft Gel Toe Stretcher and Exerciser
Zehenspanner und -trainer aus Softgel
Écarteur et correcteur d'orteils en gel souple
Soft gel teenstrekker
Межпальцевый разделитель и корректор из мягкого геля
Tåspännare och -tränare av mjukgel
Prstový korektor a posilovač z měkkého gelu
Stiradita in gel morbido

Contents/Inhalt/Inhoud/Contient:

1 pair / 1 Paar / 1 paire / 1 paar

12151

1033-M MEDO 2PK

SOFT GEL TOE STRETCHER AND EXERCISER

PolyGel® Soft Gel Toe Stretcher and Exerciser helps relieve stress and tension in your feet while helping increase circulation. The flexible soft gel is infused with 7 botanical oils and enriched with Ceramides and Aloe Vera to soothe the skin between the toes.

- Moisture infused with 7 botanical oils: avocado, olive, jojoba, grape seed, canola & soybean. Enriched with Ceramides, green tea, willow fragrance and Aloe Vera.
- Ideal for: toe pain, overlapping toes, bunions, forefoot pain, dress-shoe fatigue
- Provides healthy stretch to toes for relaxing therapy
- Massages and helps comfort sore feet
- Helps straighten and realign problem toes
- Helps stretch Achilles tendon, improve flexibility and circulation
- May alleviate condition of bunions and metatarsalgia

Directions: Beginning with the big toe, slide toes in one toe at a time. If your pinkie (smallest) toe cannot fit at first, leave it out and try to fit it in after a few uses. Toe stretcher is strong enough to stretch your toes, but soft enough that it will not make your toes hurt. We suggest wearing toe stretcher 5-10 minutes a day.

Care: Hand wash in luke-warm soapy water. Air dry. Do not use heat. Sprinkle with talcum powder if gel becomes tacky.

Material: 100% Medical Grade Mineral Oil Gel

Warning: Do not place product on an open wound. If you have diabetes, consult with your physician before use. If irritation, discomfort or poor circulation occurs, discontinue use and notify your physician. Do not put shoes on over toe stretcher. Do not walk or drive while wearing toe stretcher. KEEP OUT OF REACH OF CHILDREN AND PETS.

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