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Safety Instructions

Using instructions:

The wheelchair is thought to increase the mobility of persons who are walk - handicapped. The use of the wheelchair is dependent on your physical capacities and the handicap degree.

The wheelchair is not determined:

- To the dispensation of medicine (f. e. attaching of a drop)
- To the use in combination with other products
- for investigations or treatments (f. e. no treatment chair for tooth treatment)
- For sterilization
- to the utilization in present of anesthetizing, explosive and/or easily inflammable gases.

Security – general references:

Please read the complete operation instructions carefully before using the wheelchair the first time.

Please take care that:

- All persons that will use or repair the wheelchair will read the operation instructions
- All persons who will use or repair the wheelchair have always access to the operation instructions.

Damages that are caused by non-observance of the operation instructions are excluded of every guarantee.

Accident danger!

Do not use the wheelchair if your driving ability is restricted by medicines or alcohol.

Use the wheelchair only for its due use.

Use the wheelchair only in good technical condition.

Do not use defective wheelchairs anymore and secure them against unauthorized using.

Please remove all interferences immediately, which influence the function and security of the wheelchair.

Note the maximal load: 110 kgs

The wheelchair is only allowed to carry one person.

Danger to tip over!

Do not lean to the side over the arm support or back over the back support.

Security in driving

accident danger!!

The brake capacity of the wheelchair impairs to wet tires through rain or the travelling on damp ways.

Test the flawless function of the brakes before every use.

Secure the wheelchair in every case against rolling away by locking the parking brakes.

Use always both parking brakes.

Don't travel onto steep rises, hindrances at rise and ramps.

Do not go with the wheelchair into the water!

Danger to tip over!

Use the wheelchair only on straight, firm underground.

Avoid steep, slopes or rise.

Avoid slops or rise at which landslide danger exists (ice, snow, wetness etc.)

Avoid loose grown reasons whose state you cannot estimate (forest ground, lawn areas, beaches, gravel etc.)

Follow always the course of rise and slops – do not go zigzag.

Do not turn at rise.

Drive through hindrances and curbs always at the lowest places and rectangular.

Do not go stairways up or down.

Do not drive down rise, hindrances, stairways and curbs backwards.

Accident danger if a person pushes the wheelchair on slops

If the slops are too big it may happen, that the wheelchair cannot be held and roll down the slope uncontrolled.

Avoid areas with large slops.

Security during transportation, assembly and maintenance

If you transport the wheelchair assembled in a vehicle:


- No person may sit on the wheelchair during loading
- No one may sit on the wheelchair during transportation.

Danger of the clamping and Squeezing!

Turning parts and the fold mechanism increased danger of clamping or squeezing during the carrier preparation and during reparation:

- Perform the work to be carried out always with largest attention.
- Perform all described work only if you are sure in handling the necessary tools.
- Perform all work only with suitable tools.

Elements: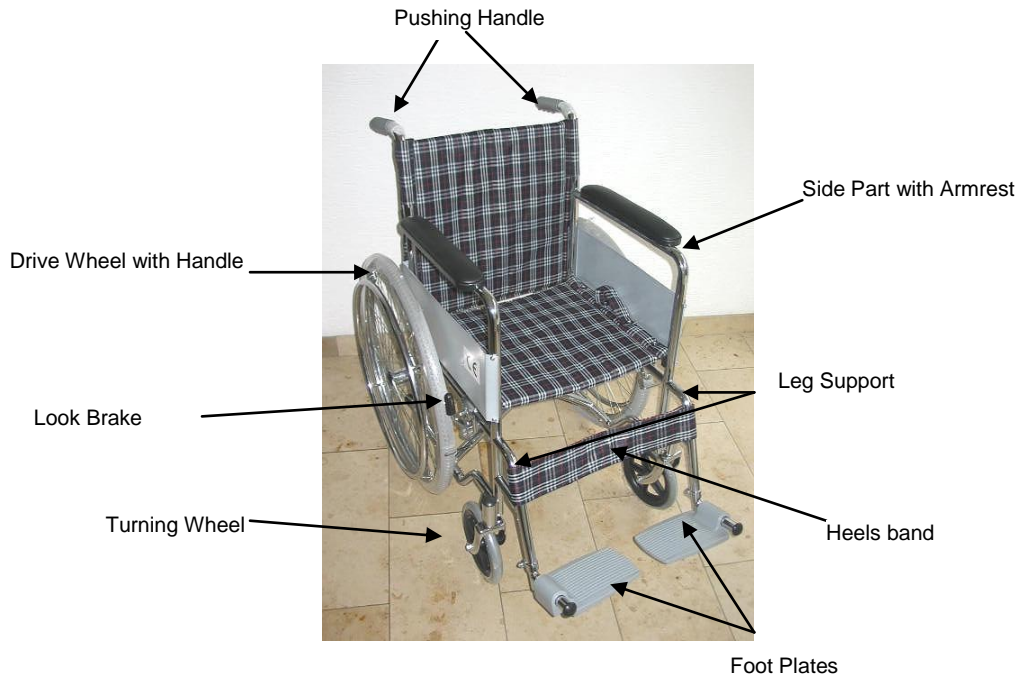

Before use:

- Remove packaging material.
- Unfold the wheelchair.
- The length of the leg support can be adjusted variably. Therefore the screw which is located at the lower end of the leg support / foot plate must be unscrewed a little bit so that the leg support tube can be pushed up or down. If you have adjusted the correct length, please tighten the screw again firmly.
- Push down the foot plates.

Fold the wheelchair:

Fold the foot plates

Pull both hand loops on the seat upwards and the wheelchair will fold.

Unfold the wheelchair:

Take both Armrests and push them apart.

Attention: Tip over the wheelchair to one side during unfolding

Please push down the wheelchair with your flat hand on the place you can see the hand loops on the seat.

Getting in and out: **accident danger!**

Before getting in or out the wheelchair make sure that the wheelchair stands on firm, even and landslide underground.

Before getting in and getting out:

Lock both parking brakes.

Use if possible a transfer help (sliding board).

Don't step on the foot plates during transfer or getting in or getting out of the wheelchair.

Push up the foot disk of the leg support.

Getting in and getting out:

Put the wheelchair as closely as possible to the transfer – object.

After you have taken a seat push down the foot plates again.

References for safe driving:

Always take care about the security references.

Driving the wheelchair is very simple and you will find it very easily after a few practices.

Please pay attention by driving:

Do not use the tires for driving your wheelchair please always use the handle rings.

Please note, that the handle rings can become smoothly moisture and you slip more easily of them.

Carry as required gloves, if necessary.

At the turning Drive-wheels increased injury danger exists.

Drive always with largest attention.

Drive always only slowly so that you can stop every time.

The parking brakes may not be used for braking.

Do not drive parallel to the stop (Danger to tip over!).

In order to avoid that the wheelchair will tip over while crossing hindrances, please note following points.

- Run over not too high hindrances.
 - Example: Curb edges
 - Remedy: Cross curb edge always at the layered entrances.
- At rises do not cross a hindrance in addition.
- Cross hindrances rectangular.
- Do not cross a hindrance with only one wheel.
- Do not go backwards over a hindrance.
- Do not drive down the curb backwards.

Secure wheelchair against rolling away !

Accident danger!

Wet tires through rain or travelling on damp ways, impairs the brake capacity of the wheelchair.

The levers of the parking brake are in the area of the wheels at both sides of the wheelchair.

Locking of the parking brake: Press lever forwards or down.

Loosing of the parking brake: Lift lever back or above until the stroke.

Retain during the standstill

In order that you will not tip over with the wheelchair in the standstill, during standstill please note following points:

- Do not grasp after objects, which lie outside of your normal range / arm length.
- Do not lean too far over the side parts and the back of the wheelchair in order to reach objects.
- Do not move forward on the seat in order to reach objects.

Insurance:

For wheelchairs, no direction to the termination of an insurance exists. Regarding liability reasons in accidents however is to be advised, to clarify, whether the wheelchair is included in the requirement case in your own liability insurance, or whether the termination of an insurance should result.

Allowance in the streets traffic:

In the sense of the streets traffic order, the wheelchair is no motor vehicle and may not participate in the streets traffic.

Use footpath.

Use pedestrian crossings for crossing streets. Take special consideration on pedestrian.

Transportation references:

The wheelchair can be folded with few steps so that it can be comfortably transported also in small vehicles.

In carrier damage, no guarantee can be received.

Transportation of the unfolded wheelchair:

Secure and fix the wheelchair with fixing straps at the vehicle so that cannot tip over.

Cleaning:

Use no tools with sharp edges or aggressive solvents for cleaning.

Use no high pressure cleaner for cleaning.

Clean all surfaces only with no abrasive and mild cleaner.

In order to avoid damages at the surfaces, take care for the user references of the cleaner.

Light dirt and dust can be removed best with a soft cloths.

Hard dirt can be removed with damp cloths and mild soap water.

Dry the wheelchair after the cleaning with a dry cloth!

Maintenance and inspection:

If you note defectives during Maintenance and inspection your wheelchair which you cannot remove firmly by yourself, contact your supplier.
Don't use defective wheelchairs any longer.

Daily maintenance before use:

Parking brakes test .
Visibility test of the handle ring, tire and rims for damages

Guarantee:

For the wheelchair, the manufacturer / importer takes over for 24 months the guarantee on flawless state in the frame of its general business conditions, excluded signs of use.

Importer: Schwartinsky GmbH, D-75015 Bretten / Germany