Premium Rollator – Artikelnummer 91167

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Premium Rollator - Article No. 91167

Instruction Manual

The modern design and sturdy construction of this Premium Rollator make it one of the safest walkers on the market, built to support weights of up to 136 kg. The unique design of the frame provides extra room for the feet to move, helping to avoid tripping and falling. It is built with the best materials available to ensure the highest safety you can find.

Our Walker
unique design
provides
extra room
for the feet to move.

Other walkers
offer less space
for the feet to move.

Other Features:

- Reinforced frame for maximum security
- Adjustable handle height
- Extra large 8" wheels for safer indoor and outdoor use
- Rubber tires provide superior grip when brakes are applied, even on smooth indoor surfaces like ceramic tiles, wood and carpet.
- Hand brake locks for safety while seated

- Comfortable extra wide backrest
- Convenient storage bag
- Convenient transport carry bag

Product Dimensions and Specifications:

- Height (min/max): 80 - 92.4 cm
- Length: 70.4 cm
- Width: 67 cm
- Seat Depth: 25 cm
- Seat Width: 46 cm
- Seat Height: 54.3 cm
- Folded Width: 35 cm
- Wheel Diameter: 20 cm
- Unit weight: 8.50 KG
- Loading capacity: 136 KG

Assembly Instructions:

1. Remove all packaging.
2. Remove the two knobs located on both sides of the walker.

3. Unfold the front and back wheels so they will be in the same position as the image shown above.
4. Once the front and back wheels are in the correct position, insert the two knobs again, making sure they go through the frame of the back wheel and the upper support of the front wheel.
5. Tighten the knobs on the opposite side of the metal frame.
6. Once all wheels are tightened and locked, unfold in the images below.

7. Lock the seat to the frame by applying pressure on both sides of the seat (see arrows on picture above) until you hear a click on each side, indicating the seat is locked to the frame.

8. Adjust the height of the handles by standing upright behind the walker with your shoulders relaxed and arms hanging at your sides. The handgrips should be at a height even with your wrists.

9. To fold the walker gently pull the nylon strap at the center of the seat. This will unlock the cross bars from the frame, allowing the walker to fold.


Operating Instructions:

To Walk

- Position yourself behind the walker and hold both handles firmly.
- Roll your walker slowly forward so that the rear wheels are a few inches ahead of your body.
- While maintaining a steady and safe roll forward, place your foot in line with the rear wheels.
- Repeat the movement with your other foot.
- To slow down, pull the brake handles up.

To Lock and Unlock Brakes

- To lock the brakes, push down on the brake handles until the brakes click into locked position, as shown below. Do not push it any further than this point.
- To unlock the brakes and resume walking, pull up the brake handles until you hear a "click."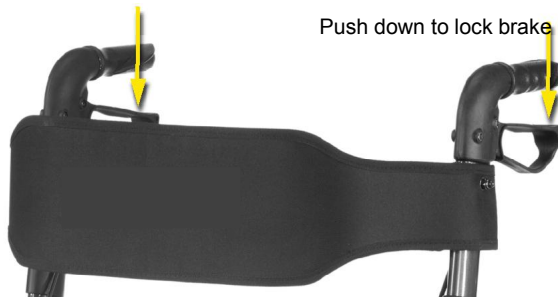

To Use as a Seat


- Make sure your walker is on a flat surface, not on an incline.
- **Lock** the brakes before sitting.
- Make sure your feet rest flat on the ground.

Care & Maintenance:

Periodic Brake Adjustment

To adjust the sensitivity of the brake:

- Turn locking nut clockwise to reduce brake sensitivity.
- Turn locking nut counter-clockwise to increase sensitivity.

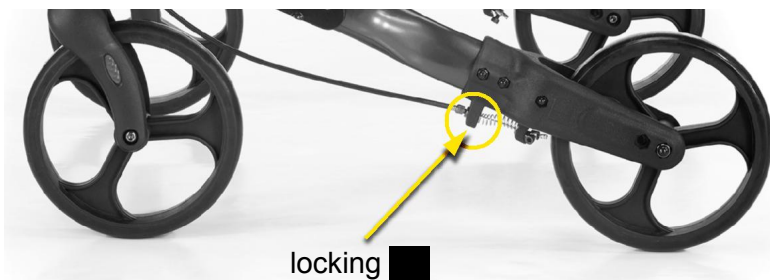WARNING!

- Over adjustment of the brake might prevent the wheels from moving freely.
- Under adjustment of the brake may cause the brake pad not to engage at all.
- ***Do not use walker if brakes are not working properly.***
- ***Do not use walker as a transport chair.***

Safety Instructions:

Warning! Failure to follow safety instructions may result in serious injury.

- **DO NOT use this product as a wheelchair.**
- **DO NOT transport someone while sitting on this walker.**
- DO NOT exceed maximum weight capacity.
- BRAKES MUST be in locked position before sitting on the seat.
- DO NOT self-propel the walker while seated.
- DO NOT sit on the walker on an incline.
- INSPECT BRAKES before each use.
- ALL WHEELS MUST be in contact with the floor or ground at all times.
- BACKREST IS NOT intended to support your full weight. It is intended to provide back support while seated.
- DO NOT use in escalators, stairs, moving sidewalks, or uneven surfaces.
- BOTH FEET should rest flat on the ground when sitting.

ATTENTION!!!

The returns will be impossible if products were damaged because of incorrect use.

DISPOSAL:

The packaging material is reusable. Dispose of the packaging in an environmentally friendly way by putting it in the appropriate recyclable waste collection containers.

Dispose of the product and the batteries in view of the environment if you want to separate. The device and the batteries must not be thrown in with domestic waste. Take them to a recycling center for used electrical and electronic devices. For more information you can consult your local government. Remove batteries first.

Eurotops Versand GmbH
Elisabeth-Selbert-Straße 3
40764 Langenfeld, Germany

If technical support is needed, please call ...

in Germany	0180 . 530 63 63* or info@eurotops.de
in Austria	0720 204 001 or info@eurotops.at
in Switzerland	071/27 4 68 06 or info@eurotops.ch
in the Netherlands	020262 1958 or info@eurotops.nl
in France	0892 . 700 470** or info@eurotops.fr
in Belgium	070 663 665*** or info@eurotops.be

* 14 Cents/minute via German landline, max. 42 cents/minute via mobile phone network

**35 Cents/minute via French landline

***30 Cents/minute via Belgium landline

A possible return of products should be addressed to the return address stated on your invoice.

© 01.09.2020