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Users manual PORTER Transportchair

General information

Thank you for having chosen a Tomtar wheelchair. The design, functionality and quality of these modern wheelchairs won't disappoint you. Before operating the unit for the very first time, please read this manual carefully.

You will obtain vital safety instructions and valuable advice on the right use and maintenance of the wheelchair. If you have questions or require additional information, please contact your specialist retailer that supplied the wheelchair.

This users manual contains all information required to adjust and operate the PORTER wheelchair.

Declaration of conformity

Dietz-Reha Produkte GmbH declares under its sole responsibility that the PORTER Transportchair described here conforms to the EU directive 93/42 for medical products.

Purpose

The PORTER Transportchair can be used both indoors and outdoors. The product is only to be used for transporting people. The wheelchair is designed for a maximum load of 100 kg. You can verify this information on the Serial number label.

Indication

Inability to walk or severe walking disability as a result of

- paralysis,
- loss of limbs,
- meromelia/limb deformation,
- joint contractures/damage to joints (not on both arms),
- other ailments.

Contraindication

Use of the wheelchair is not suited for

- inability to sit,
- use without an attending person

Signs and Symbols

Caution!

Indicates information that is particularly relevant to safety. Follow the instructions for use.

Important!

Indicates information that is particularly useful in the respective context.

Liability

DIETZ Reha-Produkte GmbH only assumes liability when products are used in accordance with the specified conditions and for the aforementioned purposes. We recommend using the products properly and maintaining them in compliance with these instructions. DIETZ GmbH Reha-Produkte does not assume liability for damage caused by components and spare parts not approved by DIETZ Reha-Produkte GmbH. Repairs may only be carried out by authorised specialist retailers or by the manufacturer.

Safety notes

In order to avoid misuse and dangerous situations, you should practice safe handling of your new wheelchair on a flat and familiar surface. Never try to move around in your Transport chair without an assisting person.

- Never leave children and youngsters unattended in the wheelchair.
- Prior to using the wheelchair, please check whether all parts attached have been fastened properly.
- The first attempts at moving the wheelchair should take place on a level and manageable surface. Familiarise yourself intensively with the braking and accelerating behaviour of the wheelchair when moving straight ahead and turning corners.
- Please note that padded parts can become hot in direct sunlight and lead to skin injury on contact. Therefore, cover these parts or protect the wheelchair from direct sunlight.
- The wheelchair may not be misused for transporting several persons or loads. Observe the specific function of the wheelchair.
- Never use your wheelchair while under the influence of alcohol or other substances that may influence your ability to react or your physical and mental receptiveness.
- Please note that redistribution of weight due to body movements or loading the wheelchair can increase the risk of tipping over.
- By all means, use ramps to avoid obstacles.
- Cross grooves, tracks and similar impassibilities in a right angle (90°), if you cannot move around them.
- There is a danger of tipping over to the rear starting at an angle of +10°.
- Avoid wheeling against obstacles (steps, kerbs) at speed or jumping down steps.
- Never stand on the foot plates to get in and out.
- Observe the road traffic regulations when on the road.
- If possible, wear bright, noticeable clothes. You will then be seen better by other traffic participants.

Product and delivery overview


Inspection of delivery

The wheelchair is completely packed in a cardboard box. Please keep the packing, if possible, it may serve for storing the wheelchair at a later time.

Upon receipt, immediately check the content for completeness and sound condition. The content consists of:

- 1 covering box
- 1 wheelchair preassembled
- 1 pair of leg supports
- 1 users manual

Features of the PORTER wheelchair

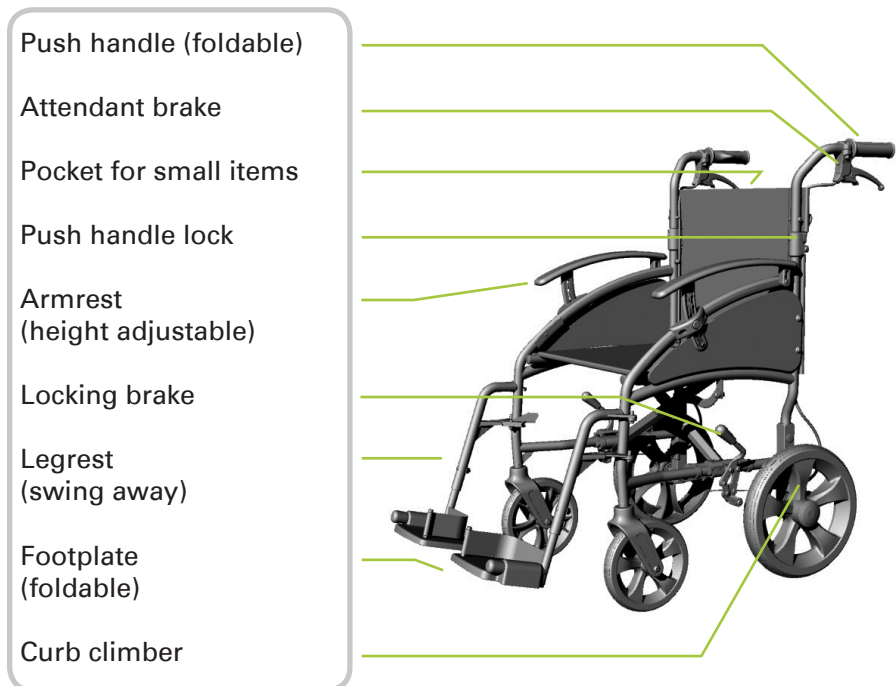Labels

Serial number and safety labels are placed on the cross-bar under the seat.

Caution!

All labels must remain readable. Immediately replace unreadable or missing labels. The serial number label is very important for identifying the product. It may not be removed.


Preparing the operating condition

Generally, your wheelchair is delivered completely assembled, folded and with the drive wheels mounted. To make it ready for use, some easy and simple hand movements are required.


Fold both push handles upwards until they lock audibly into place. Make sur both locks are properly engaged.

Position yourself behind the wheelchair and press down the crossbar with both hands.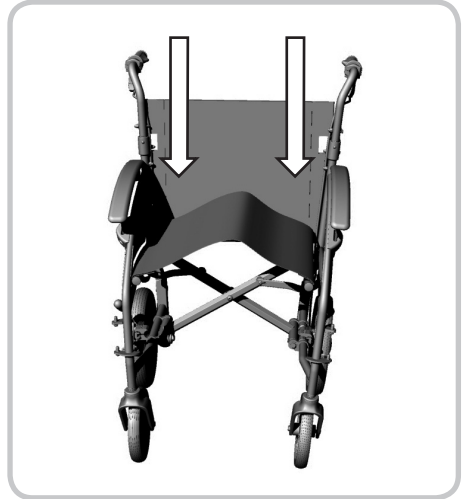

Mount the leg supports and swing them forward, they will snap in audibly.

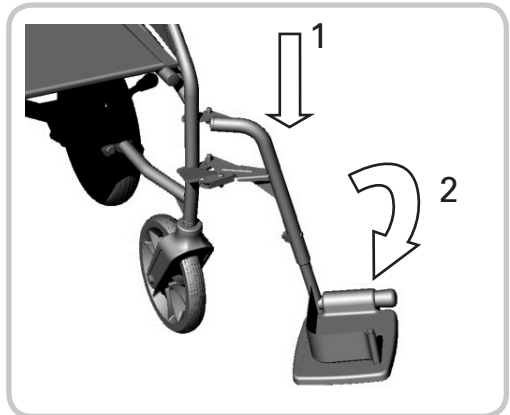Caution!
Danger of getting trapped! Watch your fingers when folding and setting up the wheelchair.

Transport and folding

For transportation in a car or simply to clear some space, the PORTER can be folded up easily. To do this, simply pull the seat upwards. The chair will now fold up all by itself. After this, you can fold down the push handles. The chair will now take very little space and can easily be stowed away.

Use of your Transport chair

Important!

The assistant must be physically and mentally capable of slowing down the wheelchair with own strength. The locking brake can be used for support.

Getting in - and out of the transport chair

The wheelchair allows you to easily get in and out. The foot plates, side parts and legrests can fold away or be retracted.

Place the wheelchair and the current or future seating surface as close beside one another as possible.

Lock both brakes.

Fold up both footplates, or alternatively swing away / take off the entire legrest.

Make sure you are sitting in the seat as far back as possible.

Finally, fold the sidepanel back down and bring the legrests back into their home position.


Caution!

Support the patient at all times while getting in - or out of the chair to avoid dangerous situations.

Armrest


Afer pushing the release button, you can move the armrests up - or downwards to find the position that suits you best. It will lock in position audibly.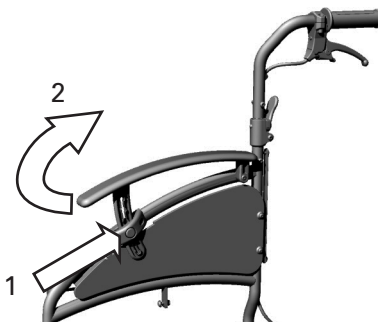

Attendant brake

To brake the chair while driving, pull both levers up evenly.


To lock the brake push the locking lever downwards. The lever is now locked.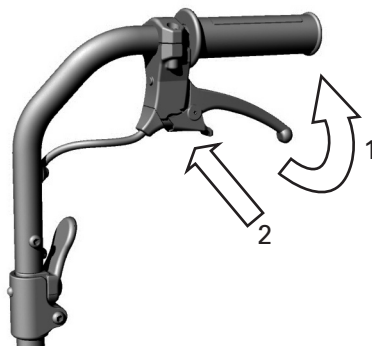

Locking brake

Alternatively you can the apply the locking brake.

The locking brake is also easily accessible for the patient while sitting in the chair.

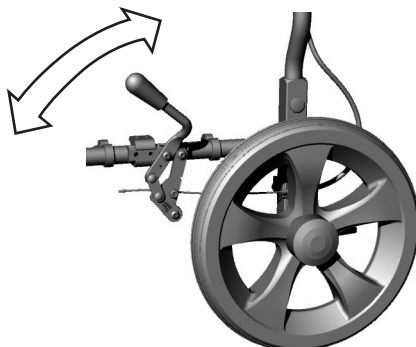Curb climber

To negotiate curbs and other obstacles you can push the curb climber gently downwards with your foot. Maintain balance with the push handles.

Adjustments

Only trained specialists should adjust brakes. However, should you have sufficient experience, you may also carry out this adjustment yourself.

Lower leg length

Loosen the screw and move the foot plate into a position that is comfortable for you. Tighten the screw again.


Brakes

With the brake fully released, the distance between the tire and the brake bolt should be ~ 17mm.

To adjust this distance loosen the 2 bolts connecting the brake body to the frame. You can now push the brake into the desired position. After this tighten the bolts again firmly and check the brakes work correctly.

You can now adjust the brake cable connecting the attendant brake lever. Finally recheck both brakes for proper function.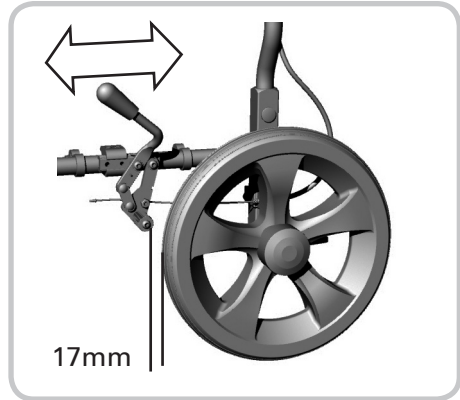

Caution!

Incorrectly adjusted brakes may result in life-endangering consequence

Advice on product care

Cleaning

- The frame parts of the wheelchair can be wiped using a damp cloth.
- In case of heavy soiling, use a mild cleaning agent. To clean the wheels use a damp brush with plastic bristles (do not use a steel brush!).
- The padding materials can be washed up to 40°C with mild soap solution and sanitised with disinfecting agents on an alcohol basis, where required.

Important!

Do not use a high-pressure or vapour cleaning device!

Transfer/further use

The wheelchair can be reused. When the wheelchair is transferred reused by another person, ensure that all technical documentation necessary for its safe use is given to the new user.

The wheelchair must be cleaned, disinfected, examined by a specialist retailer and approved.

Checking the wheelchair by the user

Prior to each use, please check:

- That the tyres are in good order and have sufficient air pressure (given optional pneumatic tyres). Check the profile depth of the tyres once a month. The profile depth should measure at least 1mm. Defective or worn tyres must be replaced.
- The operational reliability of the brakes. If the brake is in place, the wheelchair should be immovable. If the brake is released, the wheelchair should be easy to push without any grinding sounds. Should you notice any unfamiliar braking behaviour, immediately inform your specialist retailer and do not use the wheelchair any longer.

- The sound condition and cleanliness of the seat and backrest padding. If these are soiled, you can clean them in accordance with the instructions for cleaning. If they are damaged, contact your specialist retailer.
- All parts should work smoothly and not make noise. Squeaking noises indicate damage and should be examined with regard to their causes by a specialist retailer.
- That all screws and connections are tight.

Service notes for specialist retailers

We recommend having your whole wheelchair inspected by a specialist retailer once a year. In case of faults and defects on your wheelchair, return it to your specialist retailer immediately for repair.

Important!

On request, Tomtar will provide specialist retailers with required information and documents for repair and service. In case of queries, please contact the Tomtar Customer Service at phone No. +49 (0) 7248/ 9186 – 0.

Important!

Use the enclosed service record to have an overview of the service status of your wheelchair.

Disposal

If your wheelchair is not used any more and is to be disposed of, please enquire from local disposal companies about the applicable regulations.

Technical Data

Seat width cm		45	
Seat height cm	50		
Seat depth cm	40		
Seat angle °	2		
Overall width cm		64	
Overall length cm	105		
Overall height cm	97		
Back height cm	38		
Push handle height cm	95		
Armrest height cm	18 - 25		
Armrest width cm	5		
Armrest length cm	35		
Lower leg length cm	40 - 48		
Weight kg	11		
Max. gross weight kg	100		
Rear wheels	12"		
Front wheels Ø cm	20		
Tilting angle °	10		

Caution!

The wheelchair is not approved as a seat in vehicles such as disabled person transportation vehicle.

Warranty

This warranty covers all defects in the product that are shown to be caused by material or production defects. The warranty period is 24 months from receipt of the notification of readiness for despatch or from delivery at the latest. Our warranty obligations do not include damage caused by natural wear and tear, with intent, by negligent or improper operation or use. The same applies to the use of inappropriate care products, lubricants or grease. In the case of complaints, please contact your specialist retailer.

Service record

Please use, if possible, the service codes provided on the bottom of this page for filling in the columns.

when what							
Reflectors lateral							
Reflectors rear							
Drum brake							
Parking brake							
Assistant brake lever							
Push handles							
Side panels							
Seat upholstery							
Back upholstery							
Steering wheels							
Drive wheels							
Tire pressure							
Tire							
Push rim							
Frame structure							
who							

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der Dietz Reha - Produkte GmbH
Dietz Reha - Produkte GmbH