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English

1. Introduction

Thank you for having decided for the **Hydas** walking stick with step counter.

During walking with the walking stick, you will get information about your steps done, required time and calorie consumption in passing (guide numbers).

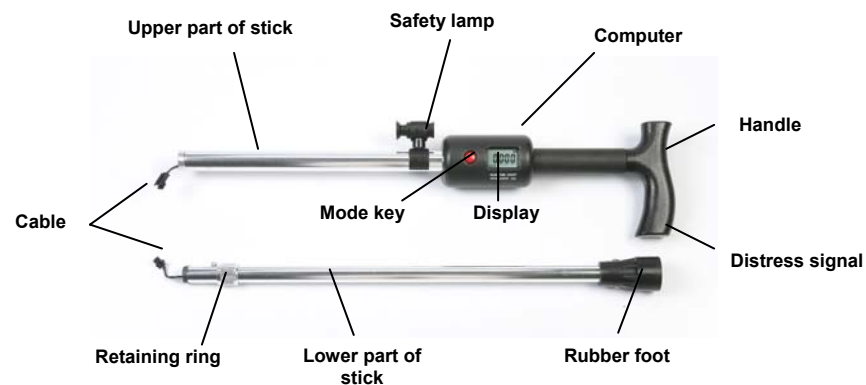
For your safety, the walking stick is provided with a distress signal and a safety lamp.

2. Main Features of the Device

Walking stick with step counter

- robust walking stick
- adjustable in height in nine levels from 79cm to 99cm
- display for:
 - number of steps walked
 - total number of steps
 - required time
 - calories consumed (guide numbers)
- distress signal in handle
- removable safety lamp adjustable as steady burning or flashing light (battery operation)
- flexible rubber foot, adjusting to floor conditions

3. Part description

4. Construction manual

Put upper and lower part of stick on an even surface area (picture A).

Pic A

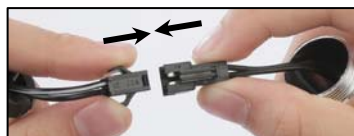
Pull cables out of upper and lower part of stick (picture B).

Pic B

Connect both plugs at cable ends (picture C).

Pic C

Put upper and lower part of stick (lower part left side) side by side.
Push retaining ring on upper part of stick in the direction of the arrow (picture D).
Tighten retaining ring in the direction of the arrow (picture E).

Pic D

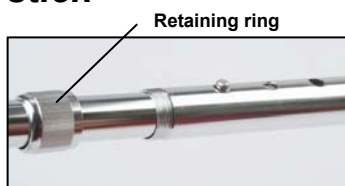
Pic E

5. Adjustable height of walking stick

For lengthening or shortening the height of the walking stick, open retaining ring in the direction of the arrow (picture F).

Pic F

To lengthen the stick, press safety key and pull lower part of stick out of the upper part of the stick in the direction of the arrow (picture G). To shorten the stick, push in the upper part of stick in the direction of the arrow (picture H).

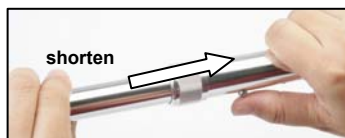
Pic G

Safety key

After height adjustment, tighten retaining ring again (against direction of the arrow, picture H).

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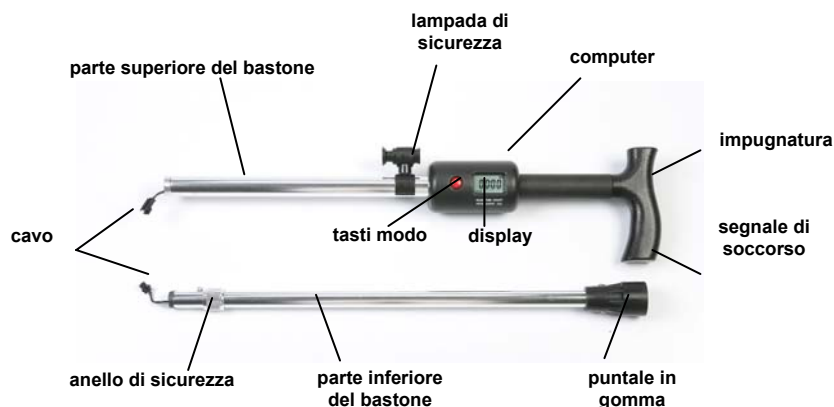
Now the walking stick is ready for use.

6. Distress signal

There is a distress signal integrated in the handle of the walking stick. In an emergency, the On/Off switch can be easily reached by the thumb. Press On/Off switch one time, the signal is switched on. If pressed again, the distress signal is switched off.

7. Safety lamp

For your safety, the walking stick is provided with a safety lamp. In an emergency or in case of darkness, the safety lamp can be switched on as a steady burning or flashing light .

For switching on the safety lamp as steady burning light, press the LED one time. Pressing LED again will switch on the flashing light.

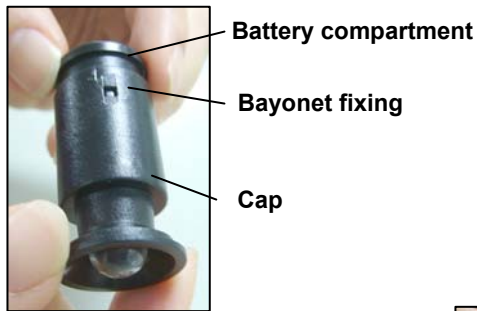
Pressing LED again will switch off the safety lamp.

The safety lamp can be turned by 360°.


8. Inserting and exchanging batteries of safety lamp:

Hold safety lamp with both hands.
Turn battery compartment as far as it will go in the direction of the arrow.
Bayonet fixing will open.

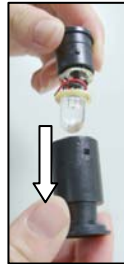Take cap off in the direction of the arrow.

Pull battery compartment out in the direction of the arrow.

Attention!

To protect the delivered batteries included in the battery compartment of the safety lamp against early discharge, there is a discharge-prevention plastic stripe in the battery compartment. Before the very first use, please open the battery compartment according to description in the directions for use and remove this discharge-prevention plastic stripe.

Operation of safety lamp requires 3 batteries á 1,5V Type LR 44.

Once the batteries are inserted or exchanged, it is proceeded in reverse order as described.

When putting together battery compartment and cap please see to it that the stud at the battery compartment fits into the guidance of the cap.

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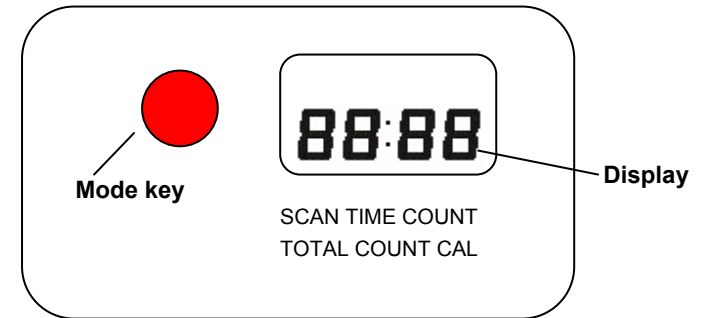
EFFECT

RESET

To reset all indicators in the display, keep the **Mode key** pressed for two seconds.

ATTENTION!

The display **Total Count** is only reset after voltage interruption (e.g. change of batteries).

STOP

After an application break of five minutes, **STOP** appears in the display.

SCAN

To display the functions, keep **mode key** pressed until **SCAN** is displayed. In the display, the functions **TIME**, **COUNT**, **TOTAL COUNT** and **CAL** are then shown for six seconds.

TIME

In **TIME mode** the time of all applications is accumulated (up to 99:59).

If the **mode key** is pressed repeatedly, **TIME** is displayed. The **TIME mode** shows the time of application (00:00 – 99:59).

COUNT

If the **mode key** is pressed again, **COUNT** is displayed. The **COUNT mode** shows the number of steps walked from 0 – 9999 during an application.

TOTAL COUNT

If the **mode key** is pressed again, **TOTAL COUNT** is displayed. The **TOTAL COUNT mode** shows the accumulated number of steps walked of all applications (0 – 9999).

CAL

If the **mode key** is pressed repeatedly, **CAL** is displayed. The **CAL mode** accumulates all calories consumed during an application up to max. 999,9 cal.

(The data ascertained are guide numbers. They must not be used for medical purposes).

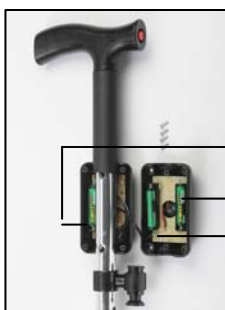
ATTENTION !

1. After an application break of four minutes the computer will automatically switch „OFF“.
2. The computer stores the ascertained data. At the beginning of a new application or if pressing the **mode key**, the values stored last are displayed.
3. If there is no clear display, please check if batteries must be exchanged.
4. Batteries for computer: 1,5V Type AAA - 3 pieces

10. Inserting and exchanging batteries of Computer:

Open screws of the lower casing by means of a Phillips screwdriver.
There are two batteries à 1.5 V AAA for the distress signal in the lower casing.
In the upper casing, there is one battery 1.5V AAA for the computer. Once the batteries are inserted, screw upper and lower part together again.

1 Battery for Computer

2 Batteries for distress signal

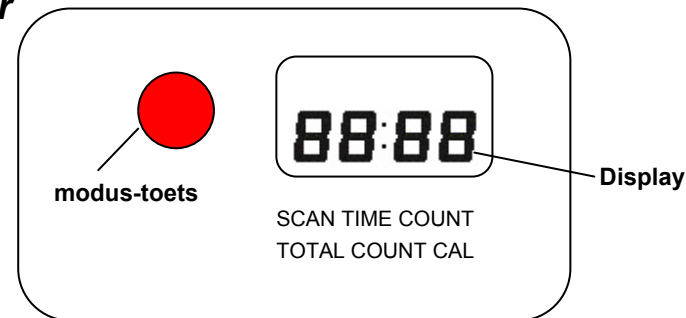
11. Package Contents

Upper part of stick
Lower part of stick
Safety lamp with fixture
Directions for use D / E / F / NL / I / E

12. Cleaning

- Clean walking stick and computer with a soft, damp cloth only. For cleaning, do not dive device in liquids.
- Use mild cleaning agents only. Do not use benzene, window cleaner, glass cleaner, furniture polish, paint thinner.

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14. Warranty:

The guarantee lasts for 24 months from the date of purchase for manufacturers' error in material and quality. This guarantee is only valid if the instructions in the "instructions for use" have been followed. The guarantee is no longer valid if the device is damaged due to violent usage, misuse or improper use or service by unqualified persons.

Address for the guarantee and service:

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Have you any questions? Hotline: 069 / 95406124

For the sake of the environment, we request that you take packaging material to be recycled (collection points).

Disposal of appliances

Electrical and electronic devices may not be disposed of with domestic waste. Consumers are obliged by law to return electrical and electronic devices at the end of their service lives to the public collecting points set up for this purpose or point of sale. Details to this are defined by the national law of the respective country. This symbol on the product, the instruction manual or the package indicates that a product is subject to these regulations. By recycling, reusing the materials or other forms of utilising old devices, you are making an important contribution to protecting our environment.

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© by **Hydas** GmbH, Am Hohlen Weg 37, D-34369 Hofgeismar, Germany