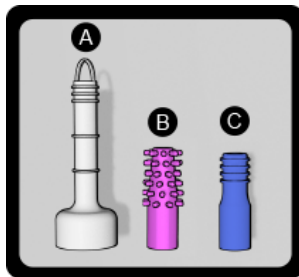
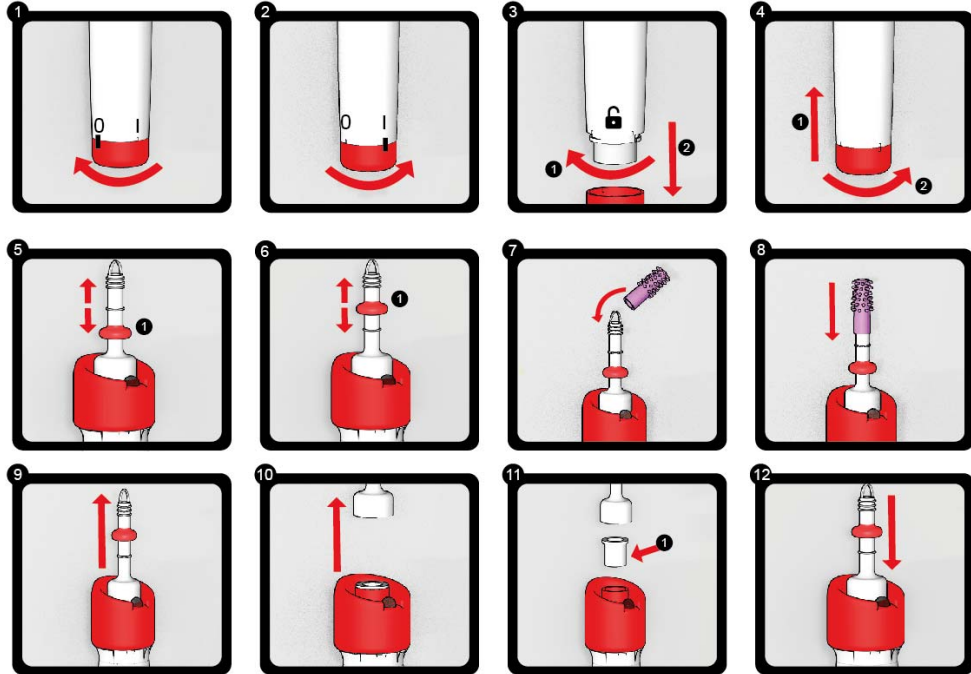
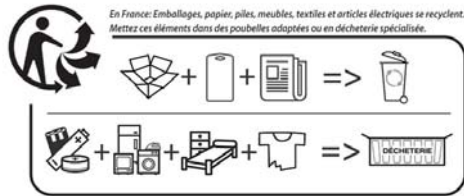
Imported by - Importé par:
GD import (Genius Ideas & Welllys)
190 chemin de la Frayère - 06530 Peymeinade - France
www.gdimport.com
www.genius-ideas.com
www.wellys-care.com

FR – ASPIR'OREILLE “SAFE EAR CLEANER” – 013927 – version 2017-10-16

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GB – SAFE EAR CLEANER – 013927

Read the operating instructions before use. Please keep it and pass on to other users.

This device allows cleaning the ear canal according to several processes.

- A- The main head is used for digging (exceptional use): make a light "scraping" of the walls. Apply with a motion from the bottom of the ear to the exit of the ear. The user must imperatively make the operation by himself, do not dig the ears of someone else (not recommended for children).
- B- Silicone tip "brush" (monthly use): put the silicone on the main nozzle tip. Introduce the nozzle in the vent and make movements of rotation of the item, with a slight back and forth movement. Tip recommended for the cleaning of the ears by a third person (recommended for children age 5).

C- Silicone tip "ringed" (daily use): put the silicone on the main nozzle tip. Introduce the nozzle in the vent and make light movements of back and forth. This cap is ideal for daily maintenance and for remove by suction water in the ears. Tip recommended for the cleaning of the ears by a third person (recommended for children age 5).

The suction is low. It allows collecting small impurities and water.

The light allows inspecting the ear canal.

Start-up and maintenance:

1. To turn on and off the unit, turn the Red base and point the spot on the O (off) or I (on).
2. In ON position, suction system and light start up.
3. To insert the battery, turn the lower part until the alignment mark on the padlock, then remove the bottom cover.
4. Place the battery into the slot, and then replace the cover and turn to lock.
5. Adjust the securing ring according to the depth of the duct. (1) An adult position.
6. Child position (1).
7. Thread the CAP silicone on the tube of the device.
8. Press down the nozzle until it stops.
9. Remove the top cap of the device.
10. Identify the filter grid.
11. Remove the filter grille, and clean it with soapy water. Dry and replace the grid.
12. Reposition the top Cap.

Recommendation:

Use only on healthy ears, non-sick, or non-swollen.

Do not use on children under 5 years.

Do not force the introduction (a too tight ear could be damaged or block the head).

Do not use the main nozzle when cleaning on a third person.

Make soft movements. Never force the eardrum.

In the event of pain or resistance, stop the use of the item.

Do not abuse of the product. Too long-term use might irritate the canal (max 1 min per day).

It is not suitable to the pain people.

Clean Head and the grid filter with soapy and water.

Avoid wetting the device. Do not immerse the unit.

Danger Associated with Batteries

- Always replace all batteries. Do not use new and used batteries at the same time. Do not use any different battery types, brands or batteries with different capacitance. When changing the batteries, check the polarity (+/-).
- Take the batteries out of the unit if they are empty or if you do not use the unit for a prolonged period of time. Thus you will prevent damages that could occur if the batteries leak.
- Non-rechargeable batteries must not be recharged.
- The terminals must not be short-circuited.

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GB - This product is labelled in accordance with European Directive 2012/19/UE concerning used electrical and electronic appliances (WEEE). Do not dispose of electrical appliances as unsorted municipal waste, use separate collection facilities. Contact your local government for information regarding the collection systems available. If electrical appliances are disposed of in landfills or dumps, hazardous substances can leak into the groundwater and get into the food chain, damaging your health and well-being.

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