

Operating Instructions for Users


How should aspUraclip med be used?

Always use **aspUraclip med** exactly according to the instructions in this guide. Please ask your doctor or pharmacist if you are not quite sure.

Clear any mucus from your nose if necessary and introduce the mini-inhaler into the nostrils – as shown on the illustration. Continue to breathe completely normally as you do so.

aspUraclip fresh is a disposable product. However, you can stop using it and continue at a later point in time. Any secretions that may stick to it should be removed with a cloth between applications. A disinfectant wipe may also be used if necessary.

Once the container has been opened, you should not use the mini-inhaler for longer than 21 days. The mini-inhaler is for your own personal use and should not be passed on to others.

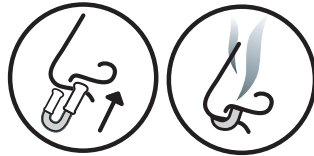

Ingredients:

The ingredient in the reservoir (2 x 0.045 ml) is a mixture of eucalyptus, thyme and ravensara oil.

Areas of application:

- To make it easier to breathe through the nose if you have a cold
- Alleviates the symptoms of cold-related sniffles and frequently recurring infections to the upper airways

Warnings:

aspUraclip med must not be used:

- If you are hypersensitive (allergic) to eucalyptus oil or one of the other ingredients.
- If you suffer from asthma or other respiratory diseases which are associated with pronounced hypersensitivity of the airways.

- If you are pregnant or breastfeeding.
- On infants and children under 6 years of age.

Can be used for up to 3 weeks

- Once the packet has been opened, **aspUraclip med** can be used for up to 21 days if used for 1–2 hours a day if it is stored in the container which is supplied with it and this container is always well sealed shut.
- The expiry date is printed on the packaging. It relates to the last day of the month. Stop using the mini-inhaler after this date.

What side-effects are possible?

- Irritations of the mucous membranes of the nose and/or eyes during use are possible. The frequency is not known.
- Hypersensitive patients have reported experiencing coughing, breathing problems (apnoea), bronchial spasms or laryngospasms. The frequency is not known.
- Interactions with other medicines have not been investigated. No interactions are known to date.
- No special precautionary measures need to be taken to be fit to drive or operate machinery.

What should I do if side-effects occur?

If you notice side-effects, first stop using the preparation and then consult your doctor if necessary. If you notice any allergic reactions in your respiratory tract (swelling on your face, mouth and/or throat area, acute shortness of breath), please consult a doctor immediately. If you notice any side-effects that are not listed in the operating instructions, please notify a doctor or pharmacist about them.

How should aspUraclip med be stored?

- Always store the mini-inhaler in the container that is supplied with it and keep it well sealed shut.
- Store it away from light and at a temperature of no more than 25 °C.
- Keep out of the reach of children.

These operating instructions were last revised in June 2018 and are available to consult online at

www.aspuraclip.com

Z 00541 In DS V1 0119