# Roomba

## Indoor-Rollator

Indoor-Rollator

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Art.Nr. / REF 730 200 100

**Teile des Rollator –**  
**Parts of walking device –**  
*Pièces du Rollator*  
 Onderdelen van de rollator

<b>1 Handgriff</b>	1 Handgrip	1 <i>Poignée</i>	1 Handgreep
<b>2 Bremsbügel</b>	2 Adjustable Tube of Handgrips	2 <i>Poignée de frein</i>	2 Rembeugel
<b>3 Feststellbremse</b>	3 Locking brake	3 <i>Dispositif de blocage du frein</i>	3 Remvergrendeling
<b>4 Clip für Höhenverstellung</b>	4 Height Adjustment clip	4 <i>Clip de réglage de la hauteur</i>	4 Klem voor hoogteverstelling
<b>5 Hinterrad</b>	5 Rear wheel	5 <i>Roue arrière</i>	5 Achterwiel
<b>6 Rahmen</b>	6 Frame	6 <i>Châssis</i>	6 Frame
<b>7 Vorderrad</b>	7 Front wheel	7 <i>Roue avant</i>	7 Voorwiel
<b>8 Korb</b>	8 Basket	8 <i>Panier</i>	8 Mand
<b>9 Tablett</b>	9 Tray	9 <i>Tablette</i>	9 Dienblad
<b>10 Bowdenzug</b>	10 Brake wire	10 <i>Câble Bowden</i>	10 Bowdenkabel
<b>11 Bremse</b>	11 Brake	11 <i>Frein</i>	11 Rem
<b>12 Tablettverriegelung</b>	12 Tray lock	12 <i>Verrouillage de la tablette</i>	12 Dienbladvergrendeling

## Indikation / Kontraindication

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### Indication / Contraindication

Walking device – for indoor and outdoor use – for compensation and improvement respectively of restricted ability of walking.

The purpose of walking devices is to compensate for or at least improve restricted walking ability in users (expansion of range and/or mobility).

Persons with restricted hand/arm functions require special adjustments/versions of handgrips or hand rests/armrests.

Impaired balance can be accommodated through appropriate safeguards on the base of the walking device.

### Installation instructions

#### Height adjustment handgrips

Take the C-cap out, draw the handle tube up and to the desire height. As Figure 1. Insert the C-cap through the hole. (see Figure 2).

NOTE: Refer to the INSTALLATION WARNINGS in the SAFETY SUMMARY of this instruction sheet.

### Opening / folding the rollator

NOTE: Refer to the FOLDING/ASSEMBLY WARNINGS in the SAFETY SUMMARY of this instruction sheet. The figures are without bag for best view.

### Opening

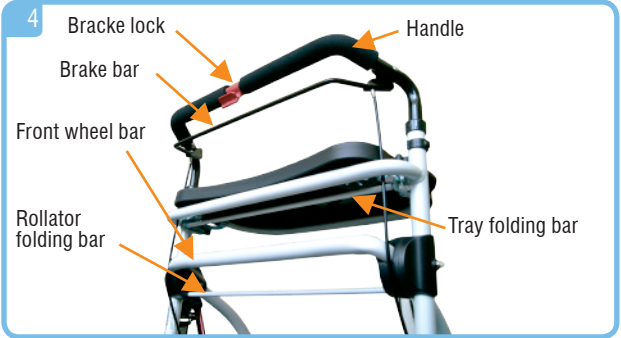
Support the front of the tray with one hand and lift the walker handle upwards, pull the tray to the direction as Figure 3. The tray is completely open with a clear “click”.

To ensure it is fully open, support the handle with one hand and press down on the tray with the other hand (Figure 3). The rollator is completely opened for use with a clear “click”.

### Folding

Grasp the tray folding bar and lift it up to release the tray and fold down. (See Figure 4).

Grasp the handle with one hand and front wheel bar center with another, drag the rollator folding bar with the left fingers to lift. When reaching the highest point, push the front wheel bar to fold it.

### Using brakes

Grasp both handles, press on the “red brake lock” to lock the rollator and it is well locked with a clear “click”. Press on the “red brake lock” to unlock and it is unlocked with a clear “click”. (See Figure 4).

### Using Tray

Ensure the tray is installed with the rollator before using. If not installed, follow the steps to re-fit it. Support the tray with both hands on the tray locks (see Parts of walking device Nr. 12) and lift them until the tray is in place, then gently press them down when the tray corners touch the columns on the bracket.

Ensure the component connects tightly on the supporting tube above the bag. Ensure the tray corner touch the columns lightly to avoid damage of the columns. Otherwise it will destroy the function of the tray locks and lift the tray lock on the bracket to unlock them. Then take the tray out by lifting up flatly.

## Maintenance

Ensure all parts of the rollator are in safe working status.

Check all screw to ensure all tubes are connected well and locked. Press and lock them if they are not. Check all the knobs and C-caps are completely inserted into the holes.

Change all damaged or worn parts timely.

## Wheels checking

Clean the wheels with water or soft cleaner and dry them with clear cloth.

Drop some 3-in-1 lube if there is any strident noise.

## Safety instructions

Use only as a walking device!

Use only on even and solid surfaces.

Walking device must not be used for sit on or as a wheelchair !It must not be used for transport of persons or animals!

The effect of the brakes can be affected by abrasion of the wheels.

Therefore test regularly the brakes.

Maximum load: 100 kg

**The use on escalator`s is expressly forbidden!**

Use tray only for small, light, but not for pointed or hot objects. Max. load: 5.0 kg

Use bag and basket only for transport of objects and do not overload. Max. load: each 5.0 kg

Do not expose the product to temperatures above 40°C for a long time, otherwise the steel pipes and the seat cover heat, which could lead to burns.

## Cleaning / Maintenance

Clean regularly by hand with a damp cloth!

Do not use any aggressive detergents!

Screws of walking device must be checked regularly and if necessary retightened.

## Reprocessing

The product is suitable for reprocessing, in line with manufacturer's recommendations.

## Type Label

1. Product name
2. Article number
3. Date of manufacturing year/month
4. Serial number
5. Attention: See operating instructions

## Warranty

We issue a warranty of 24 months from date of purchase. Subject to technical alterations. Errors and omissions excepted.

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