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CAREFULLY READ THESE INSTRUCTIONS PRIOR TO START USING THIS PRODUCT. AND KEEP IT.

GB - FOLDING WALKING STICK + "SECURITY" LAMP– 032816

This walking stick is made of Aluminium and provided with an ergonomic handle. Folds into 4 parts. It is easy to slip into a bag. Folds out automatically. The adjustable height is from 80 to 92 cm. This article is made for walking comfortably; it is not suitable for disabled, injured or sick people.

EXTENSION OR REDUCTION OF THE TUBE: Grasp the top part of the first tube of the stick with one hand and hold the lower part of it with the other while pressing the safety catch. Then adjust the height by pulling or pushing up to the desired height (maximum 92 centimetres).

FOLDING AND UNFOLDING: To fold and put away the cane please pull on each part of the rod individually in order to unlock each tube. Then use the wrist strap to tie all the parts together. To unfold the cane, just detach the strap, then place each tube of each against the other and the cane will be set up by itself. Please check the first part of the Rod including the first tube and the handle are firmly in place. **CAUTION:** Do not leave the fingers, risk of pinching!

USE:

Use of the adjustable led lamp:

First insert two batteries of type AAA into the location provided for this purpose, with the terminals (+) toward the spring (at the back of the handle). Then close the cover. To orient the lamp just direct it upwards or downwards. To turn it on or turn it off press the button at the top of the handle of the cane.

Use of audible alarm:

To turn on or turn off the audible alarm, simply press the side button on the handle.

CLEANING: Use a wet rag to clean the stick.

SAFETY WARNINGS:

Do not use on slippery or coated surfaces.

Shall Not be used by persons weighing more than 100 kg.

Do not look directly into the light.

This product has not been designed for medical use.

Do not use the walking stick if it is a disillusioned one.

Comfort article for long steps not suitable for a sick, disabled or accidental person.

WARNING: DO NOT USE THE STICK WITHOUT HAVING PREVIOUSLY CHECKED IT IS PERFECTLY FIXED AND ASSEMBLED.

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