

**Bath Tub Grab bar**

Designed for bath tubs with rim thicknesses from 7 cm up to ca. 16 cm



Dear customer,  
congratulations for purchasing your new Bath Tub Grab bar. This Grab bar makes it easier for you to getting in and out of the bath tub. It provides you more safety and comfort in the bathroom, and can be fast and easily installed at any standard bath tub.

### Safety Instructions

- The Bath tub Grab Bar is designed for private use at standard bath tubs with rim thicknesses from 72 mm to 165 mm. It is not designed for commercial use in hospitals, residential homes for the elderly, etc.
- Install the Grab bar only at the long side of a straight bath tub. The grab bar must not be used at round or semicircular bath tubs.
- The grab bar must not be used simultaneously by several persons. The maximum retention force of the grab bar is equivalent to 65 kg.
- Don't modify the grab bar in any way and don't fix any other elements on it.
- Don't install the grab bar at other furniture or fitments (e.g. table, bed, shower bath, ...)

### Instructions for Safe use

- Soap or bathing oil on your hands or on the grab bar may cause a risk of slipping off. Dry well your hands and the grab bar in before.
- Check regularly whether the grab bar is still safely fixed to the bath tub. If it is not safely tightened there is a risk that it may topple sideways when you hold on to it. Be aware of the danger of falling!
- Don't use the grab bar for storage or for fixation of bath or shower utensils (brushes, towels, etc.). These may reduce the retention force and/or may cause a trip hazard.

## Installation of the Bath Tub Grab Bar



### WARNING

Before the installation the area of the bath tub where the grab bar will be fixed must be cleaned.

This area must be clean, free from soap residues and dry, in order to ensure a safe fixation of the grab bar.



### CAUTION

Don't tighten the grab bar too tight at the bath tub rim. Otherwise the bath tub may be damaged.




### CAUTION

The grab bar is heavy. Ensure during the installation and dismantling that you do not knock against tiles or mirrors.

1. Loosen the fixation screw by turning it counterclockwise so that you can put the railing onto the rim of the bath tub. The fixation screw must face outward.
2. Then tighten the fixation screw securely by turning it clockwise.
3. Make sure that the grab bar fits securely. If you can still move the grab bar then tighten the screw more or move the railing to another position of the bath tub rim.

If you want to dismount the grab bar, e.g. for completely cleaning the bath tub, then loosen the fixation screw by turning it counterclockwise and lift the grab bar upward.

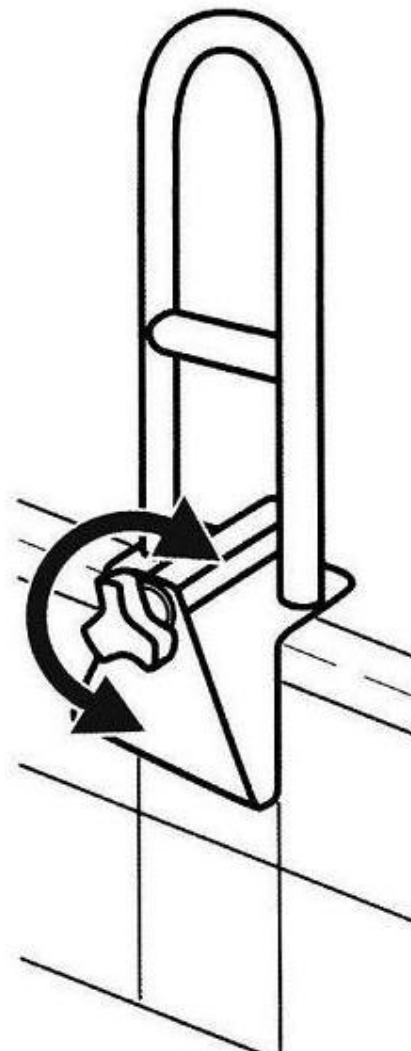

### **Cleaning and Care**

- Wipe the grab bar dry before and after each use.
- Remove any dirt with a damp cloth and if necessary with a standard household cleaning agent. Then wipe it dry.
- Dismount the grab bar for a thorough cleaning so that you can also clean the rubber pads.
- Don't use harsh cleaning agents, acids or brines.
- In case of contamination with infectious materials the railing can be cleaned with standard alcoholic disinfection liquids.

### **Maintenance**

- Regularly check the condition of the grab bar. Check especially for deformations and cracks. If you detect any defects the grab bar must not be used any more.
- If you frequently install and dismount the grab bar mind that the rubber pad is in a proper condition.

### **Instruction for Disposal**

Please dispose of the bath tub grab bar in an environment friendly way via a licensed treatment facility or via your local community facility. Please adhere to the currently valid regulations!

Model number: 304  
Article number: 06787

Weinberger GmbH & Co. KG  
Bergische Str. 11  
42781 Haan