- Ⓞ **Gebrauchsanweisung**
- Ⓞ **Instructions for use**
- Ⓞ **Mode d'emploi**
- Ⓞ **Instrucciones para el uso**
- Ⓞ **Istruzioni per l'uso**
- Ⓞ **Gebruiksaanwijzing**

- Ⓞ **Instruções de utilização**
- Ⓞ **Brugsanvisning**
- Ⓞ **Bruksanvisning**
- Ⓞ **Instrukcija obsługi**
- Ⓞ **Οδηγίες χρήσης**
- Ⓞ **Návod k použití**

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Ⓞ **LED Walking Stick & Stand Up Support**
Please read this manual carefully before use and above all observe the safety instructions! Keep instructions for future reference! The distributor does not accept any liability for damage caused by improper handling or non-observance of the safety instructions.

Insert/replace batteries:
 First insert the batteries before use. For this, remove the cover of the battery compartment (see fig. 1 ref. A) and insert 2 x 1.5 V AAA batteries (not supplied) into the battery compartment. Follow the polarity! Replace the cover.

Instructions for use:
 Please use the push button system to adjust the walking stick to the right size. When walking, hold the walking stick near to your body and always place it vertically on the floor! Before each use, check that all tube segments and screw connections are fastened and tightened properly and that the push button is properly engaged (Fig. 2 B)! Be careful on wet, icy or uneven surfaces! Make sure you have a firm hold even in the dark! If you fall nevertheless: Let the stick fall sideways to avoid falling directly on the walking stick. In case of an emergency, you can draw attention by the help of the integrated alarm function. For this, press and hold the red button on the side of the handle (Fig. 3 C) until the alarm sounds. Pressing the button again switches off the alarm. To switch on the height-adjustable LED lamp (Fig. 3 D), press the red button (Fig. 3 E) once. Press it again to activate the flashing light function on the shaft. Pressing the button again switches off both lights. Please note! The LEDs cannot be replaced!
 To use the walking stick as stand up support, fold out the lower handle (Fig. 4 F). Make sure that you stand up as vertically as possible to avoid slipping.

Safety and warning instructions:
 In order to avoid glare, please do not look directly into the switched-on LEDs!
 Not a toy! Keep out of the reach of children!
 Batteries, rechargeable batteries and electrical devices must not be disposed of with the normal household waste! All consumers are legally obliged to submit all batteries, rechargeable batteries and electrical devices,

whether they contain pollutants or not, to a collection point in their community or district or to return them to a dealer, so that they may be disposed of an environmentally compatible way. Only submit batteries and rechargeable batteries in a discharged state!

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