	FR	DE	NL	EN
①	Ecran couleur	Farbdisplay	Gekleurd scherm	Colored screen
②	Bouton de réglages	Einstelltaste	Instelbutton	Setting button
③	Touche ▲	Taste ▲	Button ▲	▲ Button
④	Touche ▼	Taste ▼	Button ▼	▼ Button
⑤	Gâchette	Auslöser	Drukknop	Trigger
⑥	Compartiment à piles	Batteriefach	Batterijvak	Battery compartment
⑦	Affichage du mode (« body » = corps / « surface » = objet)	Modusanzeige (Body = Körper / Surface = Objekt)	Weergavemodus («body» = lichaam / «surface» = materie)	Mode Display (body / surface)
⑧	Affichage de la température	Temperaturanzeige	Temperatuurweergave	Temperature display
⑨	Unité degrés C° ou F°	Temperatureinheit °C oder °F	Temperatuureenheid C° of F°	Temperature unit C° or F°
⑩	Smiley	Smiley	Smiley	Smiley
⑪	Niveau de batterie	Batteriestand	Batterijniveau	Battery charge
⑫	Logo son activé/désactivé	Ton Ein/Aus	geluidpicto Aan/Uit	Sound On/Off

THERMOMETRE FRONTAL «COLOR»

FR

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FOREHEAD THERMOMETER "COLOR"

EN

Warning: Please read this instruction manual carefully before using the instrument and keep it for future use.

A. SAFETY INSTRUCTIONS

1. The device is intended exclusively for private and not for commercial use. Only use this device as describe in this instruction manual.
2. Do not place any part of this appliance in water or any other liquid.
3. Do not handle the unit or plug with wet hands.
4. Keep it away from heat sources such as radiators, to avoid deformation of plastic parts.
5. Do not use coarse or abrasive sponges/cloths or steel wool to clean the appliance.
6. This unit is not designed to be operated by means of an external timer or a separate remote-control system.
7. It is suggested to try the device by yourself first, so as to know how to use it.
8. Do not use the infrared thermometer in extreme environments, (too cool, too high temperatures or too much humidity)
9. The person measured should be kept at room temperature for at least 20 minutes to make body temperature balanced.
10. After strenuous exercise, you should rest for at least 30 minutes before taking measurement.
11. Wipe the skin dry and push aside hair strands before commencing.
12. When repeated measurements are made continuously, move away the probe between each measurement and rest for at least 5s before making the next to obtain the most correct result.
13. Check the probe is clean before each measurement to ensure accuracy.
14. Don't touch or blow infrared sensor. A polluted or broken infrared sensor may cause inaccuracy.
15. Do not use a mobile phone nearby when the thermometer is working. Portable and mobile RF communications equipment can affect MEDICAL ELECTRICAL EQUIPMENT. Falling to do so may cause inaccuracy.
16. Do not modify this equipment without authorization of the manufacturer. The device is not repairable and contains no user serviceable parts.
17. The operator shall not touch the thermometer and the patient simultaneously.

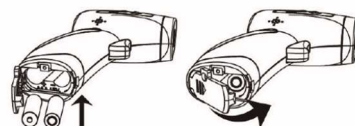
B. PRECAUTIONS D'USAGE

- Always place the thermometer in the same place on your forehead when taking a forehead temperature, as the value may be different from one place to another.
- In order to ensure the accuracy of the measurement, there should be no hair, sweat, cosmetics and dirt, etc. on the forehead when measuring.
- If you place ice or the like on the forehead of the feverish patient the measurement result will be falsified. Users should avoid action in this case.

C. HOW TO USE IT

1) Insert or replace batteries

- Remove the battery cover (6).
- Insert 2 AAA batteries observing the correct polarity indicated on the lid of the battery compartment.
- Replace the cover.
- When the battery icon "  " is indication, please replace the old battery immediately
- Please take out the battery if you do not use the thermometer for a long time.

2) Setting

- Mode selection

This thermometer has 2 different modes:

- Body mode: babies, children and adults
- Surface mode: food, water, milk etc.

Press the trigger ⑤ to turn on the device, then press the settings button ② to choose the "body" or "surface" mode.

- Temperature Adjustment

For some reason, you may want to calibrate the temperature of the unit differently than what is preset.

Turn on the unit by pressing the trigger ⑤ then press and hold the settings button ② for at least 3 seconds.

"F1" will appear on the display.

Use the arrows ③ and ④ to set the desired adjustment (between -2°C and $+2^{\circ}\text{C}$)

- Temperature alarm

By default, the temperature alarm is set to 38°C . So if the measured temperature exceeds 38°C , the thermometer will beep 3 times and the display will light up red.

To change the temperature alarm :

Turn on the unit by pressing the trigger ⑤ then press the settings button ② for 3 seconds, if you have already made the adjustment or want to skip this step, press the button ② again.

"F2" will appear on the screen.

Use the ③ and ④ buttons to set the desired alarm temperature

- Temperature unit

You can choose to display the temperature in degrees C° or F°

Turn on the unit by pressing the trigger ⑤ and then press the settings button ② for 3 seconds, if you have already made the adjustment or want to skip this step, press the button ② twice.

"F3" will appear on the screen.

Use the ③ and ④ arrows to change the unit of measurement

- Sound

You can choose to turn the sound on or off

Turn on the unit by pressing the trigger ⑤ then press the settings button ② for 3 seconds, if you have already made the adjustment or want to skip this step, press the button ② 3 times.

"F4" will appear on the screen.

Use the ③ and ④ arrows to turn the sound on or off.

3) Instruction for use « body » mode

Sometimes parents tend to rely on their principal in demand on their child's forehead, it is even a reflex whether the child is feverish or not. Or, it suffices that this hand is cold or hot to misjudge the temperature. With the forehead thermometer, the results obtained are quite precise, provided you follow the correct procedure.

You will add 0.3°C to the evaluated value to get the exact internal temperature.

Average temperature of human body is around $35,5^{\circ}\text{C} \sim 37,8^{\circ}\text{C}$. Consult a doctor if needed.

- Place the thermometer on your forehead in a vertical position at a distance of about 3 to 5 cm
- Make sure the forehead is perfectly clear and clean (no hair strands, no sweat, makeup, hat, cap etc)
- Press the trigger ⑤, the thermometer will light up and beep. The display will show the temperature.
- If you want to take the temperature again, wait at least 5 seconds between each measurement.
- To turn off the thermometer, wait at least 15 seconds.

- If a reading below 32°C is recorded a "Lo" sign will be displayed followed by 3 consecutive beeps.
- If a reading between 32°C and 37.3°C is recorded there will be 4 consecutive beeps, the screen is green color and smiley face is « happy ».
- If a reading between 37.4°C and 37.9°C is recorded there will be 4 consecutive beeps, the screen is yellow color and smiley face is « unhappy ».
- If a reading over 38°C is recorded there will be 4 consecutive beeps, the screen is red color and smiley face is « unhappy ».
- If a reading above 42.2°C is recorded a "Hi" sign will be displayed followed by 3 consecutive beeps.

It is dangerous for patients to perform a self-evaluation and self-treatment based on the measuring results. Be sure to follow doctors' instruction. A self-evaluation may cause deterioration of diseases.

4) Instruction for use « surface » mode

- Place the thermometer at a distance of about 3 to 5 cm from the target
- Press the trigger ⑤, the thermometer lights up and beeps. The display will show the temperature.

T 1303 / HG-01

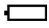
- If you want to take the temperature again, wait at least 5 seconds between each measurement.
- To turn off the thermometer, wait at least 15 seconds.

- **- If the temperature is below 0°C, the display will show "Lo" and the unit will beep 3 times consecutively.**
- **- If the temperature is between 1°C and 99.9°C, the unit will beep and display the temperature,**
- **- If the temperature is above 100 °C, the display will show "Hi" followed by 2 consecutive beeps.**

5) **Recalling Readings from Memory**

- When the unit is off, press the ⑤ trigger to turn it on.
- Press the ③ key to display and scroll through the recorded temperatures.
- If no temperatures are recorded, the display will show: " ---"
- In total the thermometer can record up to 32 measurements from 1 to 32 (1 being the oldest) in "body" mode and 32 measurements in "surface" mode.
- Each time a new temperature is taken, the oldest one is automatically erased.
- In " memory " mode, the device automatically turns off without action for more than 15 seconds.

D. **TROUBLESHOOTING**

Error message	Problem	Solution
Hi	Temperature taken is too high	Operate the thermometer only between the specified temperature ranges. In the event of a repeated error message, contact your retailer or Customer Services
Lo	Temperature taken is too low	Operate the thermometer only between the specified temperature ranges. In the event of a repeated error message, contact your retailer or Customer Services
Err	Sensor not welded	Contact your retailer or Customer Services
ErrE	EEPROM is abnormal	Contact your retailer or Customer Services
	Low power	Replace the batteries
ErH	The temperature of the measuring environment is too high.	Decrease the temperature of the environment, keep it within 16°C-35°C.
ErL	The temperature of the measuring environment is too low.	Increase the temperature of the environment, keep it within 16°C-35°C.
	Screen off	Device turns off automatically after 15seconds, please press the trigger Replace the battery

E. **CARE AND MAINTENANCE**

1) Maintenance

Remove any stains on the body with a soft and dry cloth.

Gently wipe the dirt on the body with a soft cloth, use a cotton swab or soft cloth stained with alcohol, and gently wipe the lens of the temperature measuring head. After using the product, the product can be sterilized with 75% medical alcohol on the soft towel or cotton sliver.

Don't wash with water or detergent containing abrasive or benzene.

Do not immerse in liquids.

2) Storage

The device must not be stored or used at an excessively high or low temperature or humidity, in sunlight, in association with an electrical current or in dusty locations. Otherwise may occur inaccuracies.

Remove the battery if non-use for long time.

F. **SPECIFICATION**

- Material: ABS

- Power: 2 x AAA battery

T 1303 / HG-01

- Resolution 0.1
- Operating Conditions: 10°C-40°C with a relative humidity of 15%-95%
- Storage Conditions: -20°C-+55°C with a relative humidity of 15%-95%
- Body temperature: Measurement range: 32.0°C-43°C
- Surface temperature range: 0~100°C
- Measurement accuracy: $\pm 0.3^{\circ}\text{C}$

The European directive 2012/19/EC about electronic and electric waste, requires that you can't throw away defective domestic appliance with common waste. Used device has to be collect separately to optimize the recuperation rate and the recycling of the materials to reduce the impact on the health and the environment.

This appliance is not intended for use by persons (including children) with reduced physical, sensory or mental capacities, or by persons lacking in experience or knowledge, unless they are supervised or instructed on the use of this apparatus by a person responsible for their safety. Children should be supervised to ensure they do not play with this appliance.

Precision and precautions about the battery use

- Throwing away a used battery in the environment or with household waste pollutes and avoids the recuperation of reusable materials.
- It is important to limit the battery consumption and to respect the following rules:
 - Favor alkaline battery (last longer than saline battery) and when it's possible use rechargeable battery.
 - Leave used batteries and accumulators in adapted containers that you can find in some retailers. In that way, metals don't pollute the environment and the health.
- Battery has to be inserted by respecting the polarity show on the apparel and the battery. An incorrect positioning can damage the apparel, cause leaking and even provoke fire and the explosion of the battery.
- To assure a good functioning, battery has to be in a good condition. If there is a problem, put new battery.
- Never try to recharge battery not made for this. It could leak, warm up, cause fire or explode.
- Replace all the battery at the same time. Never mix up alkaline battery with saline battery.
- Used battery has to be removed.
- Also, remove the battery if you're not using the apparel for a long time.
- Never try to short-circuit the terminal of a battery.
- Never throw away the battery in fire because they can explode.
- Take out the accumulator from the apparel before reload it.
- We recommend that an adult watch if a child change the battery to respect those rules.
- If you swallow a battery, go immediately to your doctor or the hospital.
- For the best ongoing performance, we recommend using only fresh, high-quality, alkaline batteries.

Importé par / Importiert von / Geïmporteerd door / Imported by :
CAPTELEC, 59170 Croix - FRANCE

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