SNOW

U1

SNOW Faltroller U1 Bedienungsanleitung
SNOW Folding Mobility Scooter U1 Operations Manual

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Contents

Safety Instructions	25
Product structure	26–28
Operating instructions	29–30
Instructions for unfolding your scooter	31–35
Instructions for folding your scooter	36–38
Pulling mode	39
Changing the battery	40
Technical details	41
Maintenance instructions	42
Common problems and solutions	43
Information on the warranty	44
Important information	45

Safety Instructions

The SNOW U1 is easy to use. It does not require any particular skills. You can learn how to operate it using the steps outlined on the following pages and a little practice. Read this section of the operations manual to understand the steps involved in opening and folding the scooter. Then use the manual as a reference to guide you through each step as you practise. At first it may be a little tricky, however, after some practice, it will seem easy.

ATTENTION!

Be careful when operating and take care of your hands.

Do not ride on steep slopes and take care not to tip over.

Do not let children ride alone.

Do not ride the mobility scooter in bad weather.

The SNOW U1 is a non-motorised vehicle. Do not use it on roads and be careful when riding.

Do not use it on roads with a dip of 5 cm or more to avoid possible injury or serious damage to the bodywork.

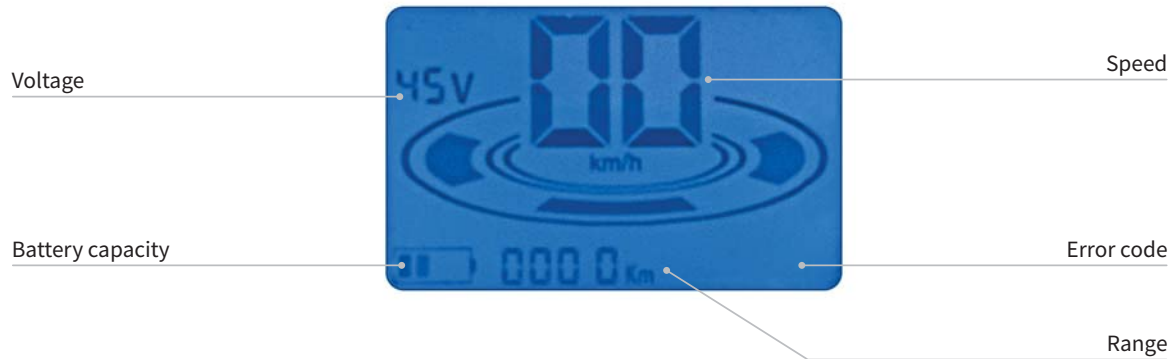
1. Product structure

2. Product structure

3. Product structure

Description of functions

Voltage: Voltage range 37 V–54 V

Speed: 6 km/h

Battery capacity: Full charge when 4 bars appear, low charge when display is flashing.

Range: Displays the total range.

1. Operating instructions

Start button: Press twice to turn on the power.

Locking switch: Press once to turn the anti-theft device on and off.

Unlocking switch: Press once to turn off the power.

Scooter search button: Press it once and the scooter will emit a sound to help you find it.

2. Operating instructions

Throttle: Press to make the scooter go.

Brake switch: Press to stop the scooter.

Horn: Press to make the scooter emit a sound to warn people in the vicinity.

Reverse switch: Press it at the same time as the throttle to make the scooter reverse.

1. Instructions for unfolding your scooter

1. Make sure that the scooter is on a level surface before unfolding it. Ensure that there is at least 2 m of space behind the rear wheel.
2. Release the angle adjustment lever (see figure 1), return the handlebars to an upright position (see figure 2) and push the angle adjustment lever back again.
3. Adjust the handlebars to an upright position.

(Figure 1)

(Figure 2)

(Figure 3)

2. Instructions for unfolding your scooter

4. Press the adjustment switch (see figure 1), hold the handlebars with one hand and bring them slowly into the position shown in figure 2.
5. Press the release switch (see figure 2), hold the handlebars with one hand and pull the folding handle with the other hand (see figure 3).

(Figure 1)

(Figure 2)

(Figure 3)

3. Instructions for unfolding your scooter

6. Follow steps 1, 2, 3 and 4. Hold the handlebars in one hand and tilt them in the direction of the height adjustment knob. Hold the folding handle with the other hand and lower it down carefully until the scooter is flat.

(Figure 1)

(Figure 2)

(Figure 3)

(Figure 4)

4. Instructions for unfolding your scooter

7. Hold the front lever (see figure 1), push down until it locks into place and a click is heard.
8. Place one hand in the recess of the backrest (see figure 2). Use your fingers to grasp the rear handle and pull the storage box up until it is at a 90-degree angle to the scooter.
9. Pull the backrest upwards (see figure 3) until it locks into place and you hear a click.

(Figure 1)

(Figure 2)

(Figure 3)

5. Instructions for unfolding your scooter

10. Loosen the height adjustment knob (see figure 1) and adjust the height of the handlebars to your preferred position.
11. Release the lever for angle adjustment (see figure 2) and adjust the angle of the handlebars to the required position.
12. The correct position when opened (see figure 3).

(Figure 1)

(Figure 2)

(Figure 3)

1. Instructions for folding your scooter

1. Press the rear handle (see figure 1) and fold down the storage box.
2. Push the storage box down (see figure 2).
3. Hold the handlebars with one hand and the front lever with the other (see figure 3). Fold the scooter together.

(Figure 1)

(Figure 2)

(Figure 3)

2. Instructions for folding your scooter

4. Follow steps 1, 2, and 3. Hold the handlebars with one hand and the folding handle with the other and pull gently upwards until you hear a click. Both sections of the mobility scooter are now together.

(Figure 1)

(Figure 2)

(Figure 3)

3. Instructions for folding your scooter

5. Loosen the height adjustment knob (see figure 1) and lower the tiller of the scooter as far as it will go.
6. Release the angle adjustment lever (see figure 2) and fold down the tiller.

(Figure 1)

(Figure 2)

Pulling mode

When you pull the scooter, let the folding handle lock into the adjustment switch.

Changing the battery

Press the button and pull the battery out.

ATTENTION!

Make sure that the battery is protected against water.

Do not use the scooter whilst the battery is charging.

Please ensure that the battery is fully charged for the first use.

Only use the original charger.

Technical details

Motor:	48 V 350 W hall brushless DC motor
Battery:	Panasonic 18650 lithium battery 48 V 5.8 Ah
Tyres:	Front: explosion-proof tubeless pneumatic tyres Back: explosion-proof tubeless pneumatic tyres
Brakes:	Electromagnetic clutches, electric brakes
Charging time:	3 hours
Weight of mobility scooter:	28 kg
Maximum speed:	6 km/h
Maximum range:	15–20 km
Gradeability:	8°
Minimum ground clearance:	95 mm
Tyre size:	Front: 8 inch Back: 10 inch
Maximum load:	100 kg

Maintenance instructions

To prolong the mobility scooter's life and make riding safer and more comfortable, please check and service the scooter regularly. Regular inspections should be carried out if the scooter is not used for extended periods of time.

If the scooter tiller starts to wobble, use an M4 Allen key, as shown in the figure above, turning it clockwise to tighten the tiller.

For long-term storage, the battery should be removed to avoid excessive discharge. During long-term storage, the battery should be fully charged and discharged every 2 months. Store the battery with a charge of 50% to conserve battery life.

Before using the vehicle on the road, please check that all components function properly. Should you notice anything unusual, please contact your dealer for maintenance or overhaul.

Common problems and solutions

Error	Error cause	Solution
The mobility scooter is on, but it is not moving.	<ol style="list-style-type: none"> 1. The battery is flat. 2. Damaged fuse 	<ol style="list-style-type: none"> 1. Charge the battery. 2. Replace the fuse.
The scooter is switched on, but the motor is not running.	The battery charge is low.	Charge the battery.
The battery won't charge.	<ol style="list-style-type: none"> 1. There is no electricity coming from the power supply. 2. Poor connection between charger plug and socket. <p>The charger is damaged.</p>	<ol style="list-style-type: none"> 1. Connect the scooter to an AC power source. 2. Check whether the socket is loose. 3. Replace the charger.
The scooter stops while driving.	The battery is flat.	Charge the battery.
Speedometer error code 00	Control error	Please take the scooter to a repair shop for a professional inspection.
Speedometer error code 01	Motor defect	Please take the scooter to a repair shop for a professional inspection.
Speedometer error code 02	Throttle defect	Please take the scooter to a repair shop for a professional inspection.
Speedometer error code 03	Low battery voltage	Laden Sie die Batterie auf.

Information on the warranty

The warranty is not valid in the following circumstances:

1. There is no valid warranty certificate, customer service record, sales receipt or invoice.
2. The user disassembles, modifies, repairs or damages the normal condition of the product and its components.
3. The product is faulty or damaged due to human error by the user, e.g. by collision, overcharging or overspeed.
4. Damage caused by improper use by the user.
5. Damage to the sold products due to rain, fire, chemical corrosion or force majeure.
6. For wearing parts (such as tyres).

Important information

Please note, inside the box there is a USB stick. On it, there is a video explaining the folding mechanism. Simply insert the memory stick into a USB connection port on your PC. This will also work when the computer is already on. Windows will automatically recognise the memory stick as a so-called removable media. Now you can click on the file that is on the stick and the video window will open.

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SNOW by ECONELO GmbH · Im Brühl 34 · 74348 Lauffen am Neckar