## **Gebrauchsanweisung (D)**

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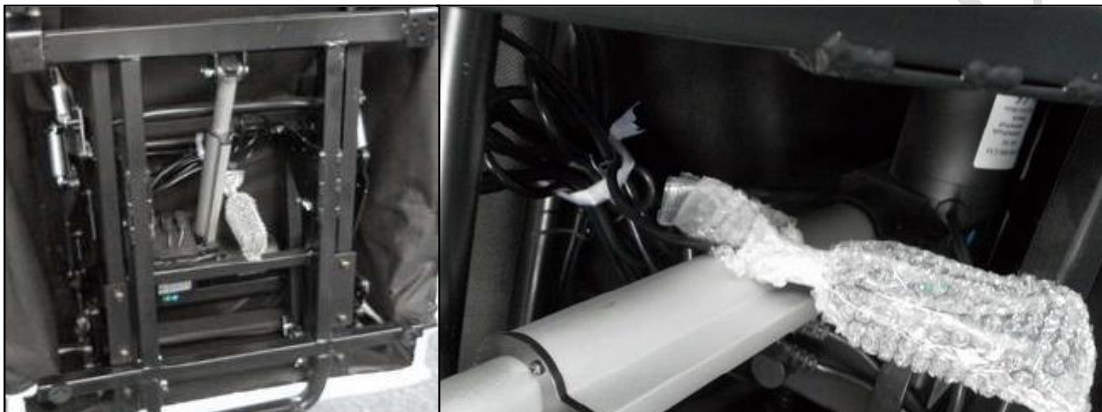
# GB

## Remark

1. The motor extensions wires are simply fixed on the rear rail of the pedestal.
2. The power cable is already connected and fixed together with the remote controls of actuator (lifting and reclining) on the bottom of the relax chair (s. Images). Bend the relax chair and detach the fastening with 2 min. persons. Don't use a knife or scissors for detaching the cable insulation to avoid damaging on the isolation.

3. Pull the remote control cables out to the rear so that they cannot be pinched or damaged. For improper handling and disregard of the instructions, as well as the resulting damages, no warranty claims can be asserted!

### **Connecting the remote control for Lift - up**

The connection for the remote control for Lift-up located under the chair (is slightly thicker than the power cord connection)

#### **Remote control for the lift function, massage function and lying position**

- Press and hold the down arrow key to activate the lift function.
- Press and hold the up arrow key to bring the chair into the normal sitting position.
- Press and hold the up arrow key to move the chair to the lying position and lift the footrest.
- Press and hold the down arrow button to return from the lying position to the sitting position.
  
- **Massage:**
- The 4-step massage function is activated by the ROUND button.
- Adjustable by repeatedly pressing the ROUND button

#### **IMPORTANT SAFETY INSTRUCTIONS**

When using an electrical appliance, basic precautions should always be followed, including following:

1. Never sit or lean on the remote control and never place heavy objects on the remote control.
2. Do not put the remote control on the floor.
3. Do not drag the cables forcefully.
4. Never use the remote control with wet hands.
5. Do not touch the buttons with sharp objects.
6. Do not unplug the remote control without turning it off before.
7. This product is for indoor use only.
8. Always unplug the chair from the power supply after use and before cleaning.
9. Close supervision is necessary when this product is used by or near children, invalids or disabled persons.
10. Use this product only for its intended use as described in this manual.
11. Do not use attachments not provided with this unit.
12. Do not sit on the arm rest.

13. Do not expose any part of the chair to direct sun light or heat sources.

#### Health cautions

1. If you have any concerns regarding your health, consult your doctor before using this product.
2. If you experience pain in muscle of joint for a prolonged period in time, discontinue using and consult your doctor. Persistent pain could be a symptom of a more serious condition.

#### Care and Maintenance

1. To Store:
  - keep the unit in a clean and dry condition.
  - for long time storage, cover the chair up to protect from dust.
  - keep the unit away from direct sunlight or high temperature.
2. To clean:
  - make sure the unit is switched off and unplugged from power before cleaning.
  - clean the cover by wiping down with a damp cloth and using a mild detergent of suds. Pat dry with clean and dry cloth.
  - do not use abrasive cleaners, brushes, gasoline, kerosene, glass/furniture polish, paint thinner or water hotter than 48°C to clean the unit.
  - use a clean and dry cloth to clean the remote control.

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